



Depth Stop .085 I.D.

INSTRUCTIONS FOR USE

Table of Contents

- I. Intended Use**
- II. Device Description**
- III. General Warnings and Precautions**
- IV. Use Instructions**

I. Intended Use

The MR Compatible Device Depth Stop is intended as an accessory for medical devices that may need depth limits. The device is not intended for implant. This device is intended for “single patient use only”.

Warning: This device is intended for “single patient use only”. Contents of unopened, undamaged package are sterile. Do not re-sterilize.

II. Device Description**REF NGS-DS-06: Depth Stop (6pk)**

The Depth Stop consists of two components: a ring and a set screw, both made from plastic. The package contains 6 total Depth Stops.

III. General Warnings and Precautions

Warning: Do not use if the package or any of the parts of the device are damaged.














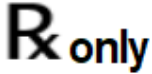
Warning: There are no known and reliable means of cleaning, disinfecting, and sterilizing these devices that returns them to original specifications and renders them safe and effective for reuse.

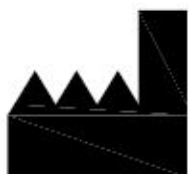
Caution: The compatibility of instruments/devices with the Device Depth Stop should be evaluated before use. The components are MR safe.

Caution: Handle all components using standard hospital sterile practices. Store in cool dry place.

IV. Use Instructions

1. Remove the device from the package. The Depth Stop is packaged in a double sterile barrier; two sealed mylar/Tyvek pouches.
2. Place the Depth Stop onto the device/instrument.
3. Carefully tighten the Depth Stop to prevent damage to the other device/instrument.

SYMBOL	DEFINITION	SYMBOL	DEFINITION
	MR Safe		Sterilized using ethylene oxide
	Consult instructions for use		Manufacturer
	Catalogue number		Keep dry
	Batch code		Keep away from sunlight
	Use by date		Single use
	Double sterile barrier system		Do not use if the product sterilization barrier or its packaging is compromised
	Not made with natural rubber latex		Prescription Device

**Manufactured by:**

ClearPoint Neuro, Inc.
 6349 Paseo Del Lago
 Carlsbad, CA 92011
 USA

949-900-6833