SMARTGrid Planning Grid

INSTRUCTIONS FOR USE

Table of Contents

- I. **Intended Use**
 - II. **Device Description**
 - III. **General Warnings and Precautions**
 - IV. **Use Instructions**
 - A. **Preparation**
 - B. **Marking the Burr Hole Site**
- I. **Intended Use:** The SMARTGrid is an accessory to the ClearPoint System to assist in providing stereotactic planning, guidance and delivery of devices within the brain.
- II. **Device Description:**

Package Contents:

NGS-SG-01-11: SMARTGrid Planning Grid

Marking Grid (1), Marking Tool (1)

This is a MR visible planning tool for surgical point of entry. The SMARTGrid Marking Tool is MR Conditional.

SMARTGrid

Marking Tool
Herramienta de marcado
Outil de marquage
Strumento di
Markierungs-Tool
Narzędzie do znakowania
Mærkningsværktøj
Markeergereedschap
Markeringsverktøy
Ferramenta de Marcação

III. General Warnings and Precautions

- a. **Warning:** This device is intended for single patient use only. Contents of unopened, undamaged package are sterile. Do not re-sterilize.
- b. **Warning:** There are no known and reliable means of cleaning, disinfecting, repairing, and sterilizing these devices that returns them to original specifications and renders them safe and effective for reuse.

- c. **Warning:** Do not use if the package or any of the parts of the device are damaged.
- d. **Caution:** Federal Law restricts this device to sale by or on the order of a physician.

Note: Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and appropriate Competent Authority of the Member State in which the incident occurred.

Use Instructions:

A. Preparation

1. Position and secure patient in an appropriate head fixation frame and appropriate imaging coils to achieve desired image quality and provide access to the procedure site.
2. Install a sterile field drape and attach the incise tape portion of the drape to the patient's head.

B. Marking Burr Hole Location

1. Remove paper back from Marking Grid.
2. Affix sticky side of Marking Grid onto the patient's head (over the incise tape portion of the drape), centered at the approximate location for the burr-hole.
3. Determine location on SmartGrid to mark skull per instructions from CLEARPOINT workstation (alpha-numeric coordinates).
4. Peel off top fluid filled layer of SmartGrid while firmly holding the lower layer in position.
5. Using Marking Tool and the coordinates, mark location on skull, entering through one of the holes provided on the Grid. The Marking Tool should be twisted/rotated several times using sufficient pressure to get a recognizable mark on the skull.

Caution: Only use minimum force required to leave a reference mark on the skull. This is only meant to provide surgeon a landmark, not to penetrate patient's skull.
Peel off remaining bottom layer of SMARTGrid.

Note: Safe disposal of the device: The device shall be treated as biohazardous materials and shall be disposed of accordingly per hospital policy.

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Beekley Corporation
ClearPoint Neuro, Inc.