# CLEARPOINT

## NEURO

**Inflexion SMS Sola-Vida Head Fixation Frame - BR**

**INSTRUCTIONS FOR USE - INSTALLATION**

## Table of Contents

1. Device Description
2. Use Instructions
  - A. Head Fixation Frame Installation

	<b>NOTE:</b> These installation instructions are to be used in conjunction with the Universal Inflexion Head Fixation Frame Instructions for Use
---	--

### 1. Device Description

#### Package Contents:

#### **CP-INHFF-132-01-BR: ClearPoint Inflexion SMS Sola-Vida Head Fixation Frame**

Inflexion Head Fixation Frame Base, Lower Ring (1), Upper Ring (1) *(optional)*, Posts (6), Short Fixation Screws (4), Medium Fixation Screws (4), Long Fixation Screws (4), Extra Long Fixation Screws (4), Post screws (8) Green or Black Knob (4), Yellow Knob (4), Screw Tool (2), Screw Tool Crossbar (2), Hex Knob (2).

#### Table Configurations

Fits Siemens Sola-Vida - MR Table configurations.

#### Associated Devices

#### **NGS-SP-01: Skull Pin Tips**

MR Compatible Skull Pin Tips

#### **NGS-SP-02: Pediatric Skull Pin Tips**

MR Compatible Skull Pin Tips, Pediatric

**Warning:** This device is intended for multiple uses. Contents are non-sterile.

**Caution:** Federal (U.S.) law restricts this device to sale by or on the order of a physician.

**Note:** The Head Fixation Frame is **MR Conditional**

The Inflexion Head Fixation Frame is designed to immobilize the patient's head during the surgical procedure. The patient's head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure. Positioning of the head may be performed remotely with the Arc separated from the Base or with the Arc attached to the Base.

The Inflexion SMS Sola-Vida Head Fixation Frame is compatible with the following MRI Systems:

- SMS – Sola-Vida 1.5T and 3T. **See Figure 1.**

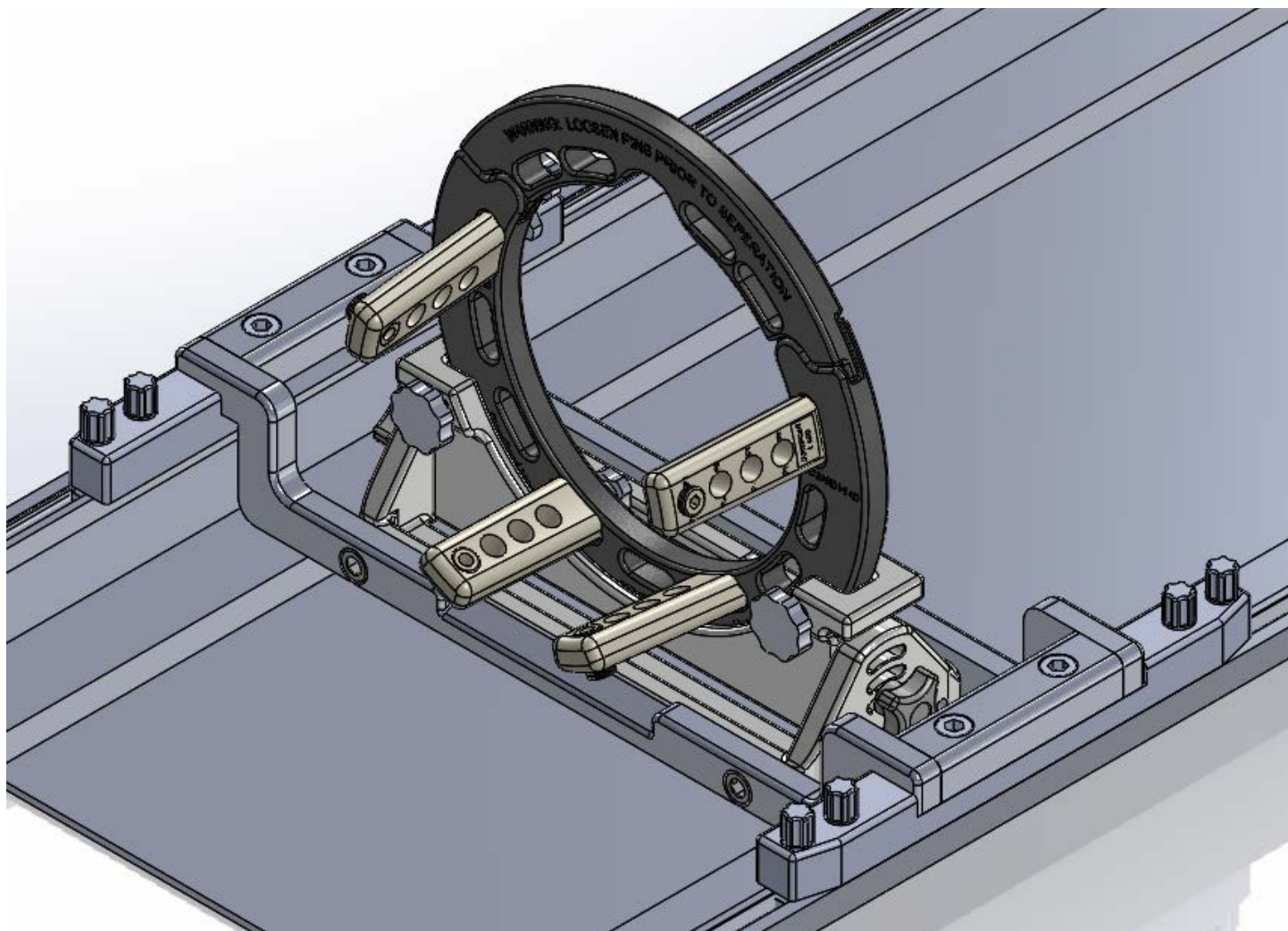
Figure 1: Head Fixation Frame with Table Mounts

2. Use Instructions

A. Head Fixation Frame Installation

1. The cradle has a "d" shaped groove on each side as seen in the top view in Figure 2a and end view in Figure 2b. There are also cut outs seen in Figure 2a that the SMS Sola Plug Screw will fit in. **See Figures 2a and 2b.**

Figure 2a (Top View)

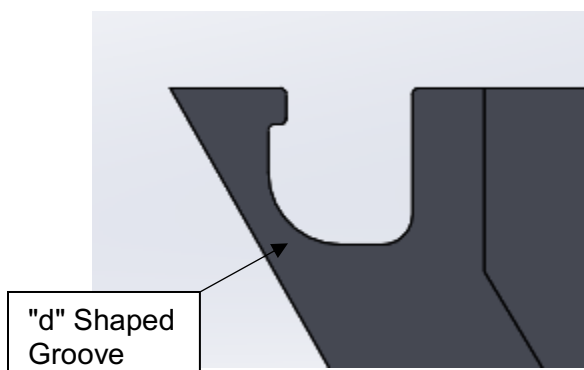
Figure 2b (End View)

2. Remove the 8 SMS Sola Plug Screw and place them down into the "d" groove aligning them with the 2 Table Mounts (one on each side). **See Figure 3.**

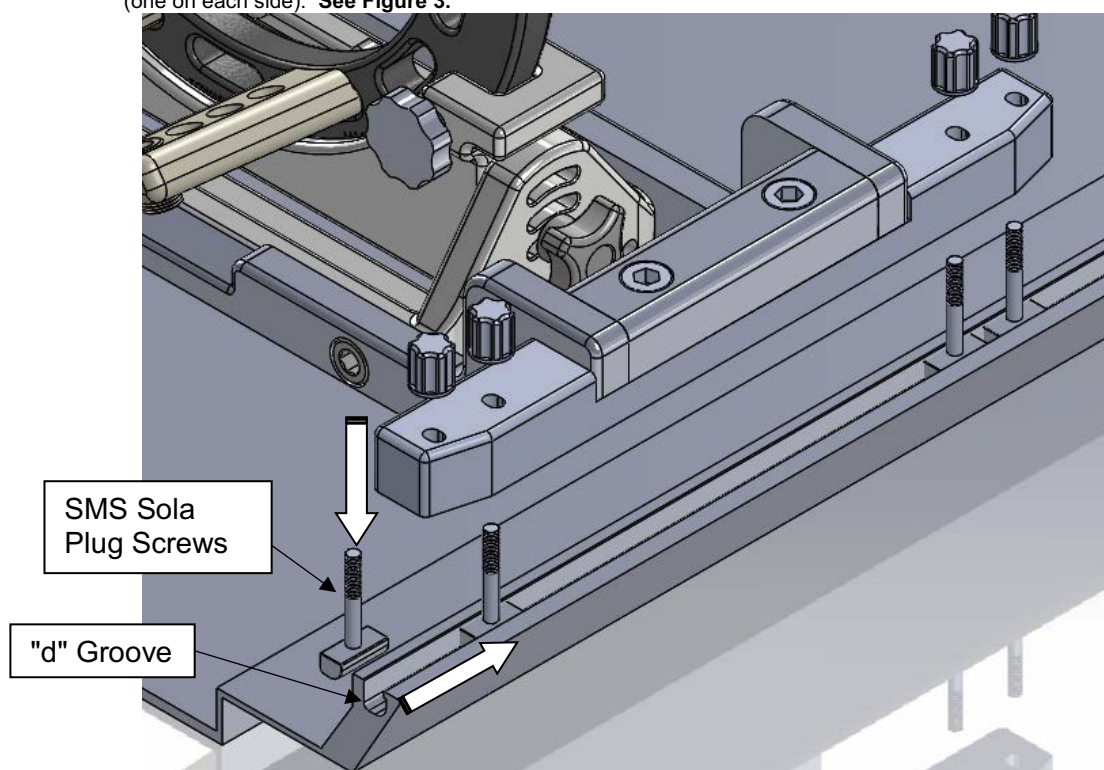
Figure 3

3. Once all 8 SMS Sola Plug Screws are aligned with the 2 Table Mounts – Lower the Head Frame down onto the Cradle. **See Figure 4.**

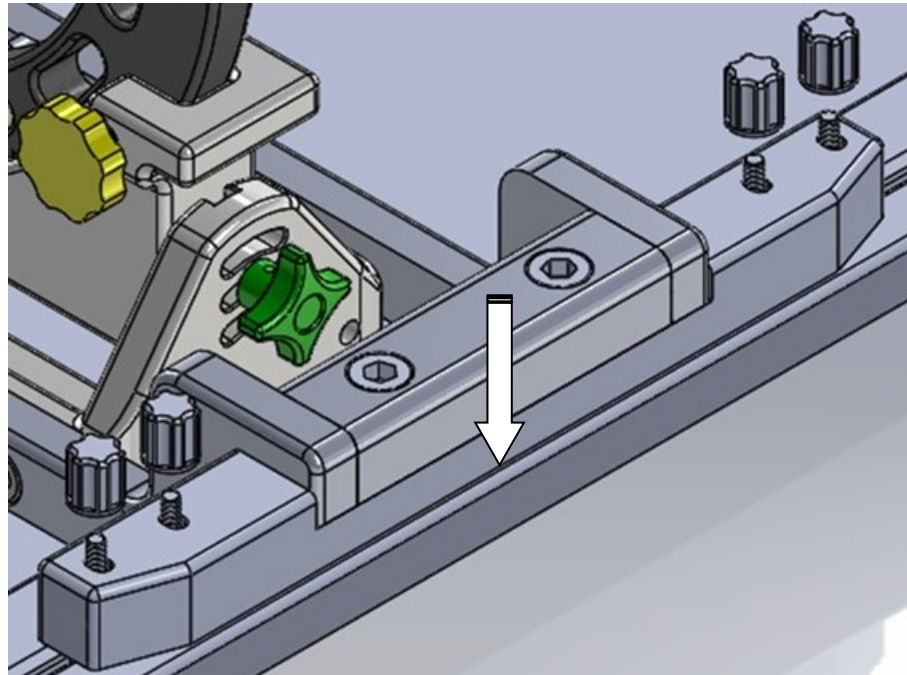
Figure 4

4. Fasten the Head Frame to the Cradle using the 8 Knobs. **See Figure 5.**  
DO NOT OVER TIGHTEN

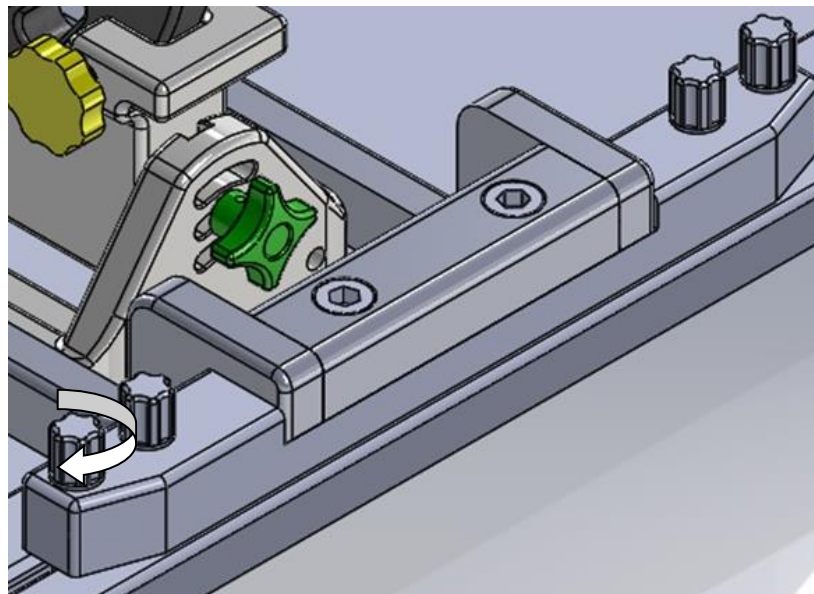
Figure 5

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ClearPoint Neuro, Inc