**CLEARPOINT**<sup>®</sup>  
NEURO

**Inflexion GE HDw-450w Head Fixation Frame**

**INSTRUCTIONS FOR USE – INSTALLATION [EN]**

**CE**

**Table of Contents**

- 1. Device Description**
- 2. Use Instructions**
  - A. Head Fixation Frame Installation**

	<p><b>NOTE: These installation instructions are to be used in conjunction with the Universal Inflexion Head Fixation Frame Instructions for Use.</b></p>
---	--

**1. Device Description**

**Package Contents:**

**CP-INHFF-322-01: ClearPoint Inflexion GE HDw-450w Head Fixation Frame**

Inflexion Head Fixation Frame Base, Lower Ring (1), Upper Ring (1) *(optional)*, Posts (6), Short Fixation Screws (4), Medium Fixation Screws (4), Long Fixation Screws (4), Extra Long Fixation Screws (4), Post screws (8) Green or Black Knob (4), Yellow Knob (4), Screw Tool (2), Screw Tool Crossbar (2), Hex Knob (2).

**Table Configurations**

Fits GE HDw-450w - MR Table configurations.

**Associated Devices**

**NGS-SP-01: Skull Pin Tips**

MR Compatible Skull Pin Tips

**NGS-SP-02: Pediatric Skull Pin Tips**

MR Compatible Skull Pin Tips, Pediatric

**Warning:** This device is intended for multiple uses. Contents are non-sterile.

**Caution:** Federal (U.S.) law restricts this device to sale by or on the order of a physician.

Note: The Head Fixation Frame is **MR Conditional**

The Inflexion Head Fixation Frame is designed to immobilize the patient's head during the surgical procedure. The patient's head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure. Positioning of the head may be performed remotely with the Arc separated from the Base or with the Arc attached to the Base.

The Inflexion GE HDw-450w Head Fixation Frame is compatible with the following MRI Systems:

- GE HDw-450w 1.5T and 3T. **See Figure 1.**

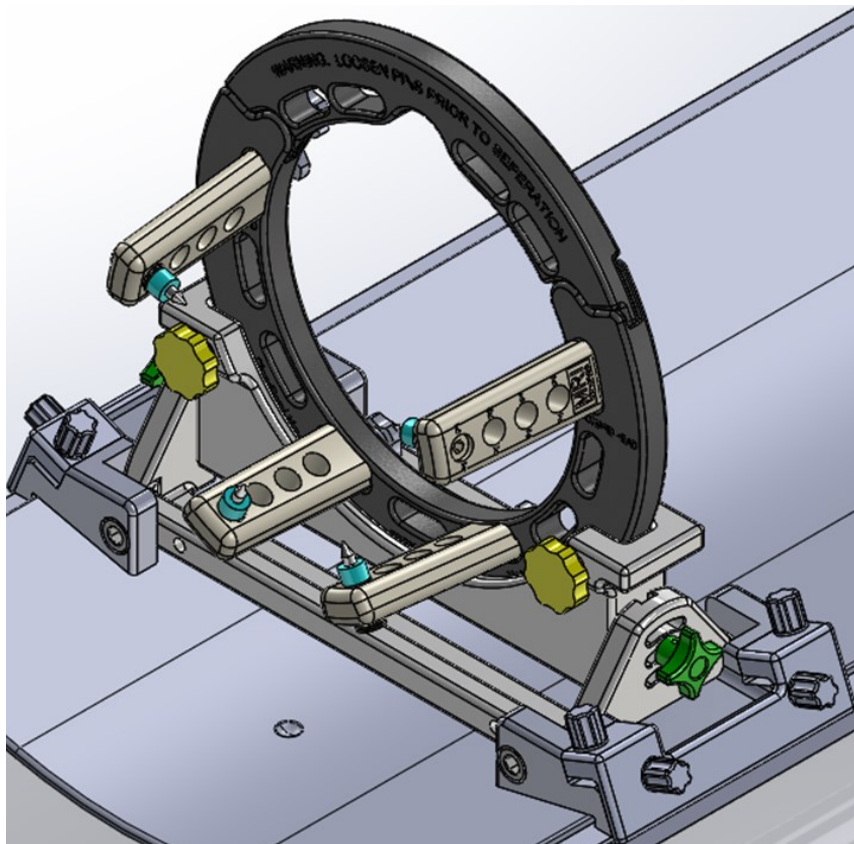
Figure 1 Head Fixation Frame with Table Mounts

2. Use Instructions

A Head Fixation Frame Installation

1. The cradle has a groove on each side running the full length of the cradle.
2. At the superior end of the cradle there is a cut-out / pocket on each side of the cradle. See Figure 2.

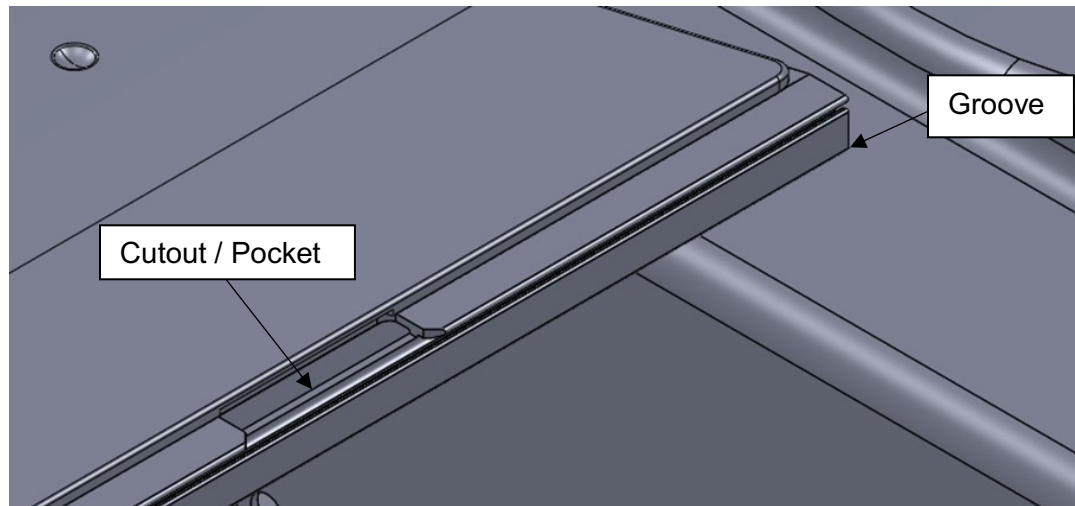
Figure 2

3. Loosen the eight (8) Thumb Screws of the Head Fixation Frame. Four on each side. **See Figure 3.**

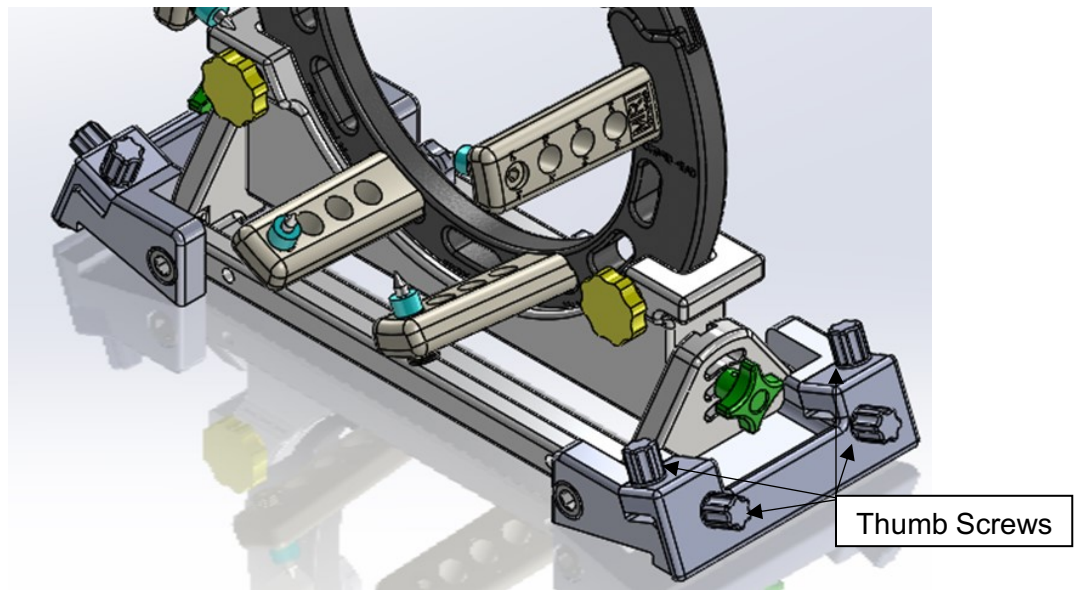
Figure 3

4. The bottom of the Head Fixation Frame has a Toe and a Lip on each side of the Frame. **See Figure 4.**

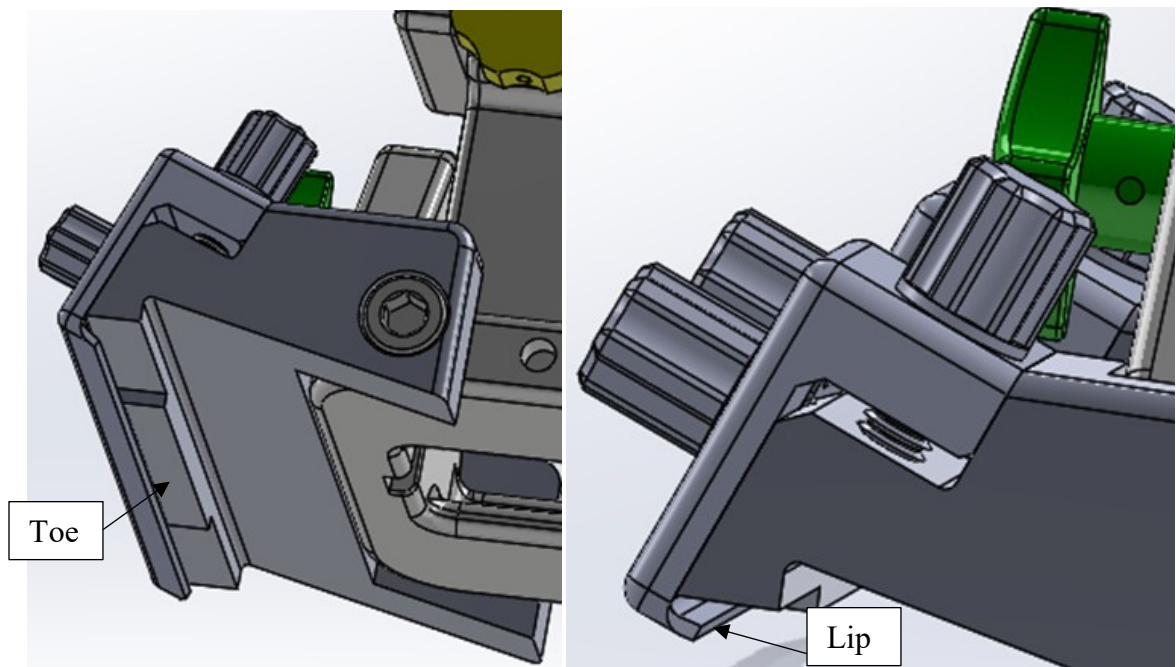
Figure 4

5. Place the Head Fixation Frame onto the cradle with the Toes inside the cut-out / pockets and the Lips into the grooves on each side of the cradle. **See Figure 5.**

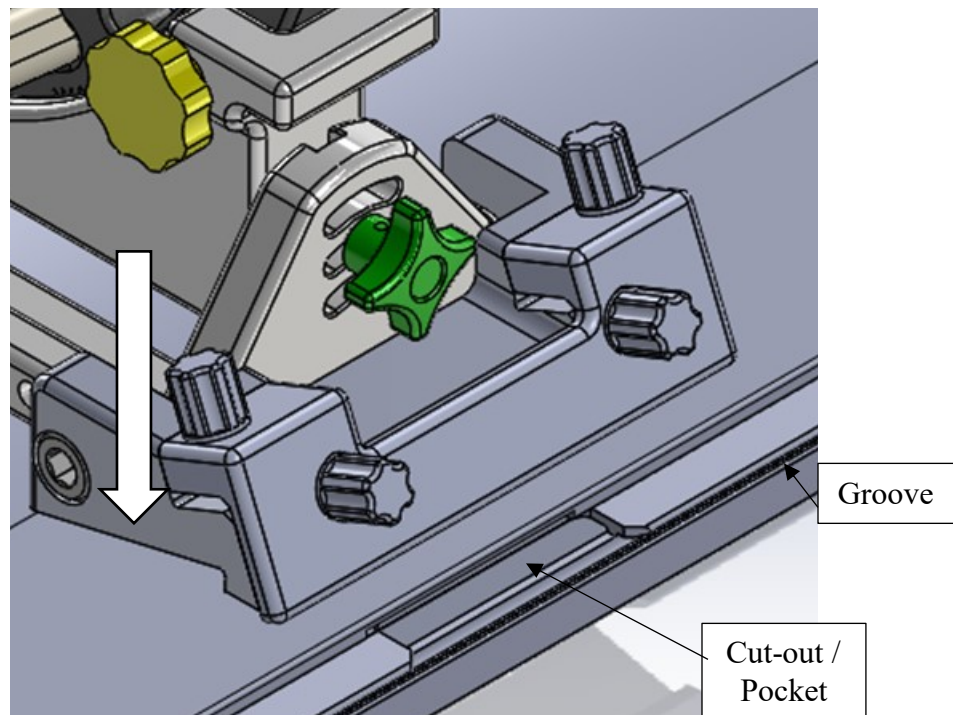
Figure 5

6. 1) Tighten the side Thumbscrews first (leave them slightly loose). 2) Tighten the top Thumbscrews until the Head Fixation Frame is secured to the cradle. 3) Now tighten the side Thumbscrews until the Head Fixation Frame is secured to the cradle. Repeat this on the other side. **See Figure 6.**

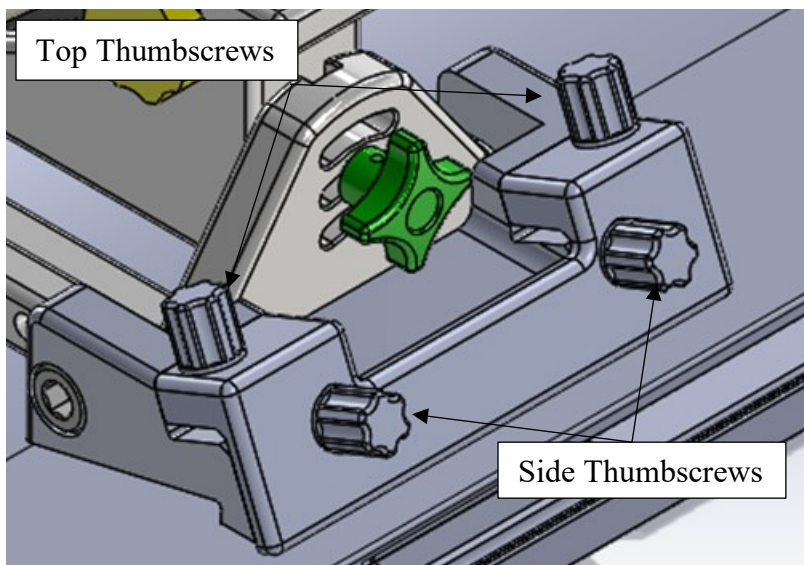
Figure 6

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ClearPoint Neuro, Inc.