# CLEARPOINT<sup>®</sup>

## NEURO

**Inflexion Philips Ingenia Head Fixation Frame**

**INSTRUCTIONS FOR USE – INSTALLATION [EN]**

**Table of Contents**

- 1. **Device Description**
- 2. **Use Instructions**
  - A. **Head Fixation Frame Installation**

	<p><b>NOTE: These installation instructions are to be used in conjunction with the Universal Inflexion Head Fixation Frame Instructions for Use.</b></p>
---	--

1. **Device Description**

**Package Contents:**

**CP-INHFF-422-01: ClearPoint Inflexion Philips Ingenia Head Fixation Frame**

Inflexion Head Fixation Frame Base, Lower Ring (1), Upper Ring (1) (*optional*), Posts (6), Short Fixation Screws (4), Medium Fixation Screws (4), Long Fixation Screws (4), Extra Long Fixation Screws (4), Post screws (8) Green or Black Knob (4), Yellow Knob (4), Screw Tool (2), Screw Tool Crossbar (2), Hex Knob (2).

**Table Configurations**

Fits Philips Ingenia - MR Table configurations.

**Associated Devices**

**NGS-SP-01: Skull Pin Tips**

MR Compatible Skull Pin Tips

**NGS-SP-02: Pediatric Skull Pin Tips**

MR Compatible Skull Pin Tips, Pediatric

**Warning:** This device is intended for multiple uses. Contents are non-sterile.

**Caution:** Federal (U.S.) law restricts this device to sale by or on the order of a physician.


Note: The Head Fixation Frame is **MR Conditional**

The Inflexion Head Fixation Frame is designed to immobilize the patient's head during the surgical procedure. The patient's head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure. Positioning of the head may be performed remotely with the Arc separated from the Base or with the Arc attached to the Base.

The Inflexion Philips Ingenia Head Fixation Frame is compatible with the following MRI Systems:

- Philips Ingenia 1.5T and 3T. **See Figure 1.**

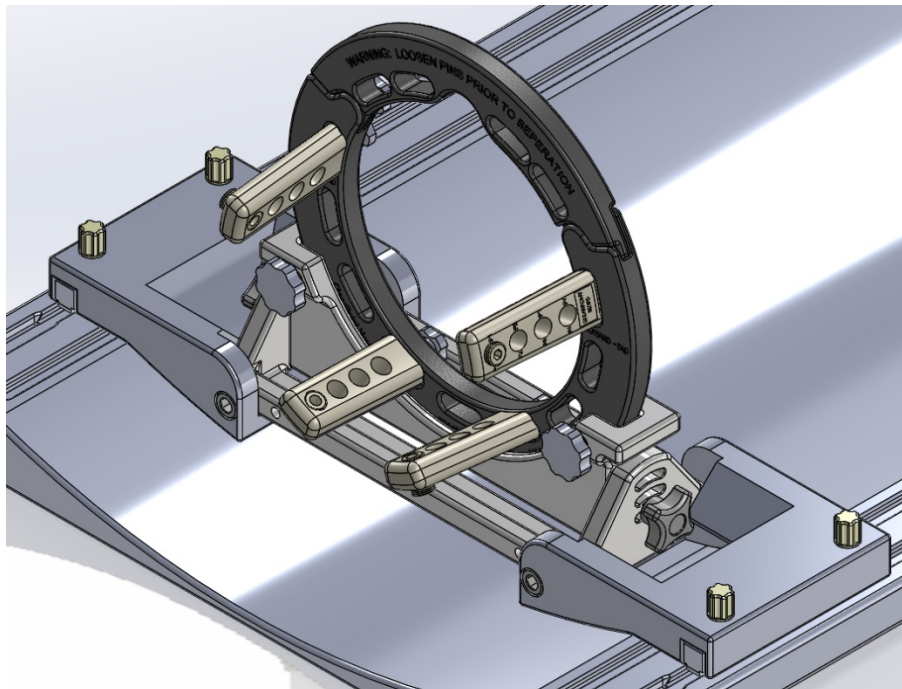
**Figure 1 Head Fixation Frame with Table Mounts**

## 2. Use Instructions

### A Head Fixation Frame Installation

1. The cradle has a groove on each side running the full length of the cradle.
2. At the superior end of the cradle there is a cut out on the outer groove only, one side each. Note: The Philips Ingenia Head Fixation Frame is designed to fit the outer groove with the cut outs. **See Figure 2.**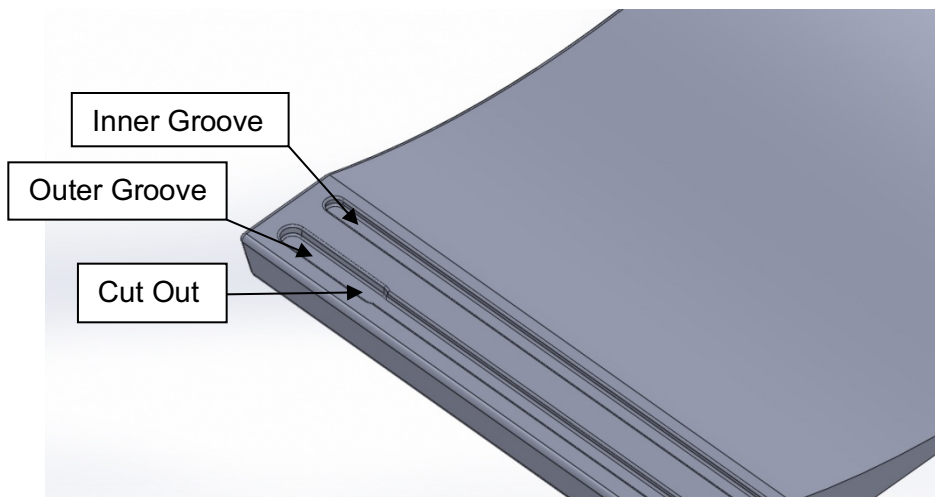

**Figure 2**

3. Loosen the four (4) Thumb Screws of the Head Fixation Frame. **See Figure 3.**

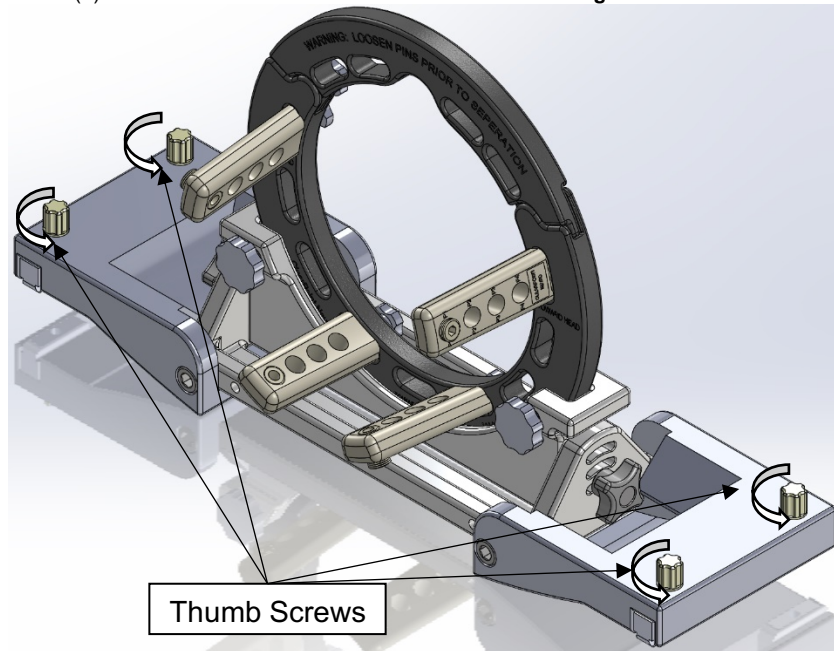
Figure 3

4. Install the Head Fixation Frame onto the MRI Table, by setting the Dovetails into the cut-outs at the end of the table. **See Figure 4.**

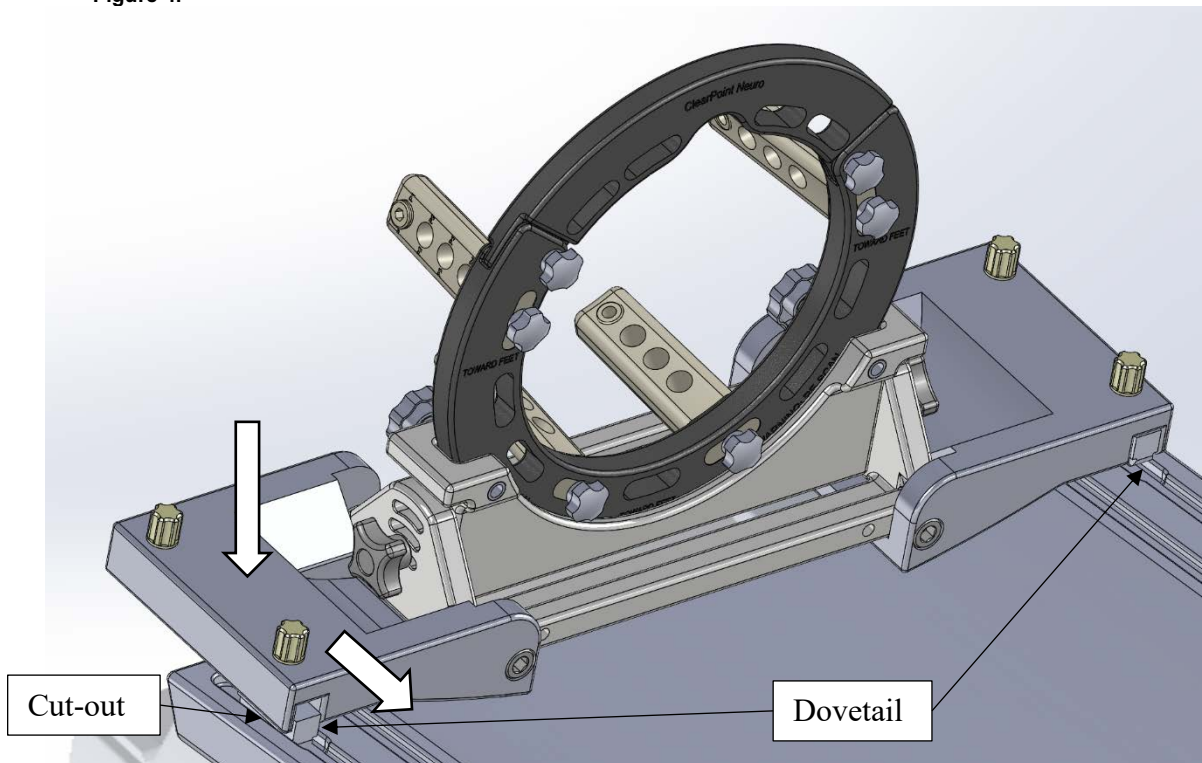
Figure 4

5. Gently slide the Head Fixation Frame forward enough that all four Dovetails can be fully engaged, and the Head Fixation Frame is to the desired position. See Figure 5.

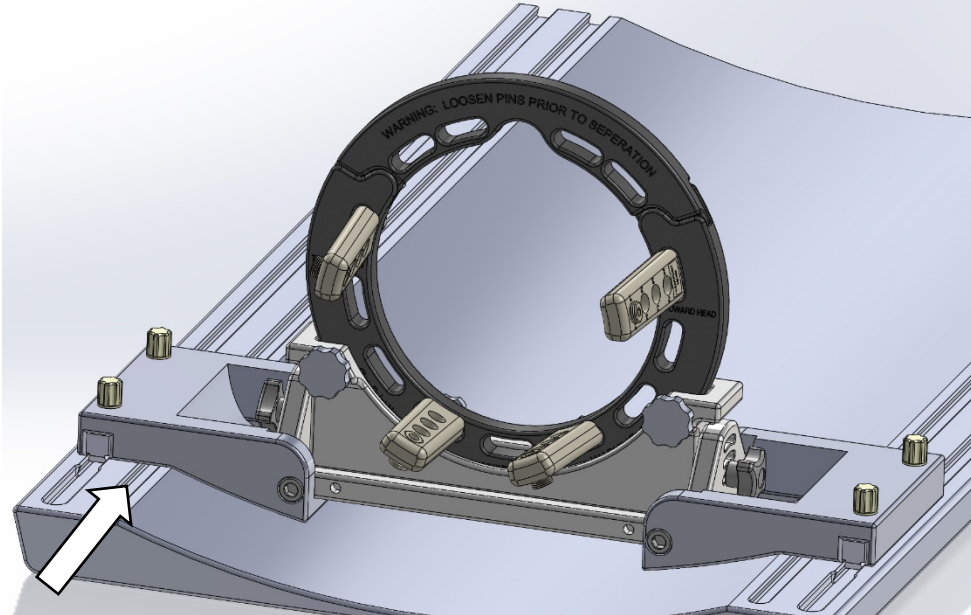
Figure 5

6. Gently tighten the Thumb Screws to secure the Fixation Frame to the Table. Note: It is very important not to over tighten the Thumb Screws, over tightening can damage the MRI Table. See Figure 6.

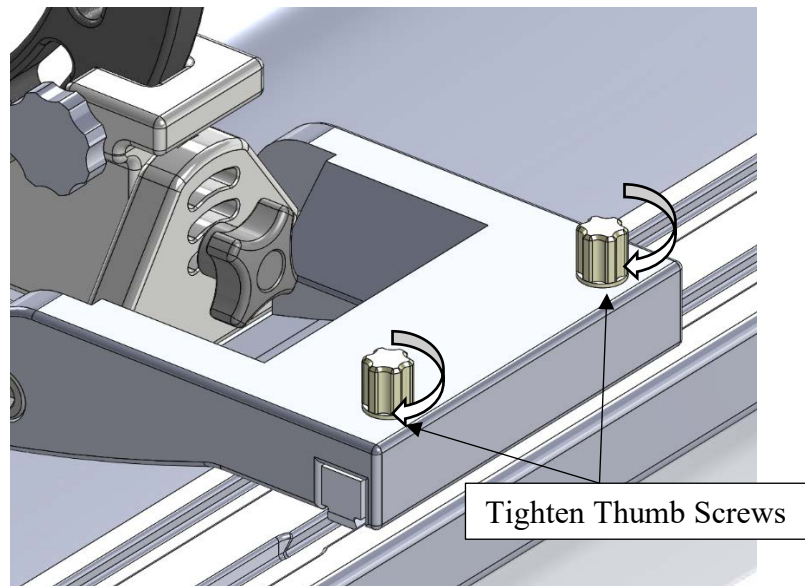
Figure 6

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ClearPoint Neuro, Inc.