# CLEARPOINT<sup>®</sup>

## NEURO

**SMS - Aera/Skyra Multi-Positional Head Fixation Frame**

**INSTRUCTIONS FOR USE – INSTALLATION [EN]**

**NOTE: These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use**

**CE**

## Table of Contents

**1. Device Description**

**2. Use Instructions**

**A. Head Fixation Frame Installation**

	<p><b>NOTE:</b> These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use.</p>
---	---

**1. Device Description**

**Package Contents:**

**CP-MPHFF-122-01: ClearPoint Multi-Positional Head Fixation Frame (New Built)**  
 (SMS – Aera/Skyra – MR Table Compatible)

Head Fixation Frame, Upper Fixation Screws (6), Lower Fixation Screws (4),  
 Drive Wheels (2)

**-OR-**

**CP-MPHFF-123-01: ClearPoint Multi-Positional Head Fixation Frame (Superior)**  
 (SMS – Aera/Skyra – MR Table Compatible)

Head Fixation Frame, Upper Fixation Screws (6), Lower Fixation Screws (4),  
 Drive Wheels (2)

**Associated Devices:**

**CP-FFSMPU-01: Upper Fixation Frame Screws (replacements in case of damage)**  
 Upper Fixation Screws (6)


**CP-FFSMPL-01: Fixation Frame Screws for GE, Siemens and IMRIS (replacements in case of damage)**  
 Lower Fixation Screws (4)

- CP-FFSMPL-02: Fixation Frame Screws for Philips (replacements in case of damage)**  
Lower Fixation Screws (4)
- NGS-SP-01: Skull Pin Tips**  
MR Compatible Skull Pin Tips
- NGS-SP-02: Skull Pin Tips**  
MR Compatible Skull Pin Tips, Pediatric

The Multi-Positional Head Fixation Frame is designed to immobilize the patient's head during the surgical procedure. The patient's head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure.

The Multi-Positional Head Fixation Frame is compatible with the following MRI Systems:


- SMS - Aera/Skyra 1.5T and 3T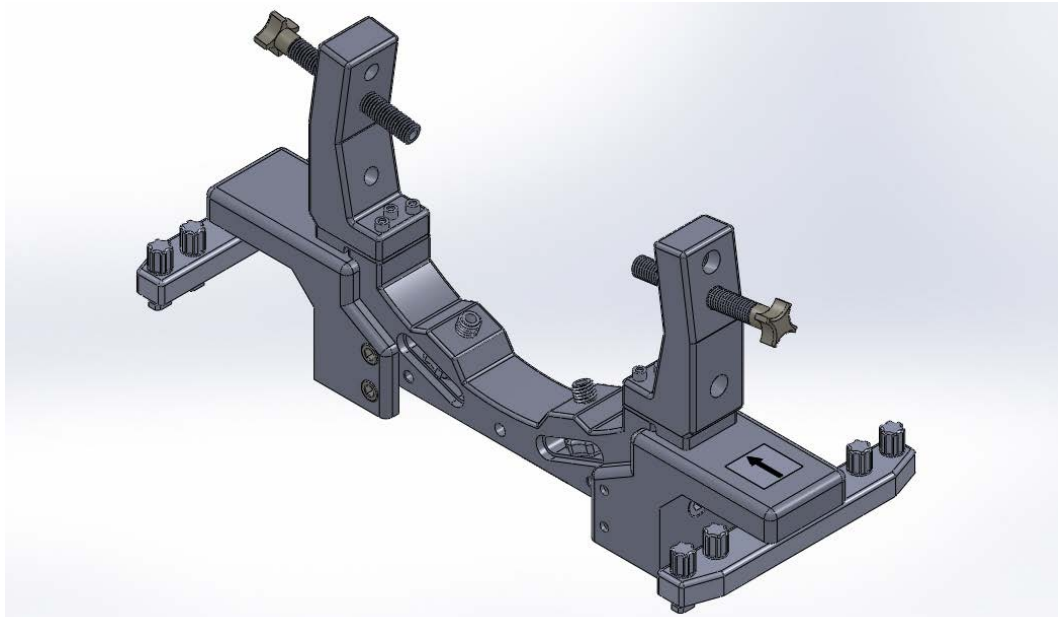


## 2. Use Instructions

### A. Head Fixation Frame Installation


1. The Cradle has a "T" shaped groove on each side. There is also cut outs that the T-Screws will fit in.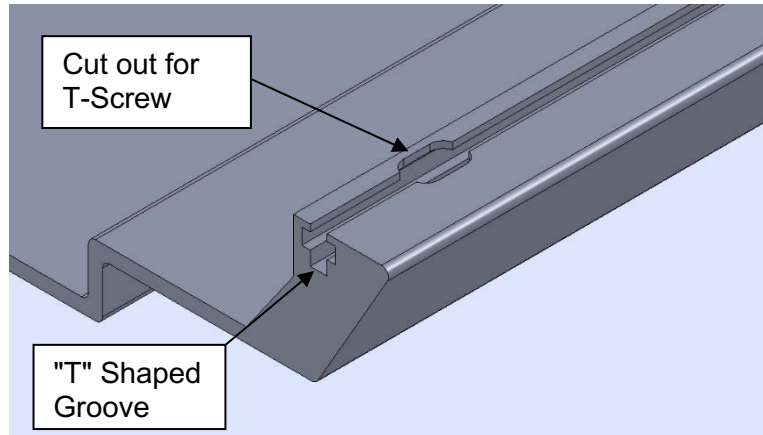

2. Remove the 8ea T-Screws and place them down into the "T" Groove aligning them with the 2ea T-bars (one on each side).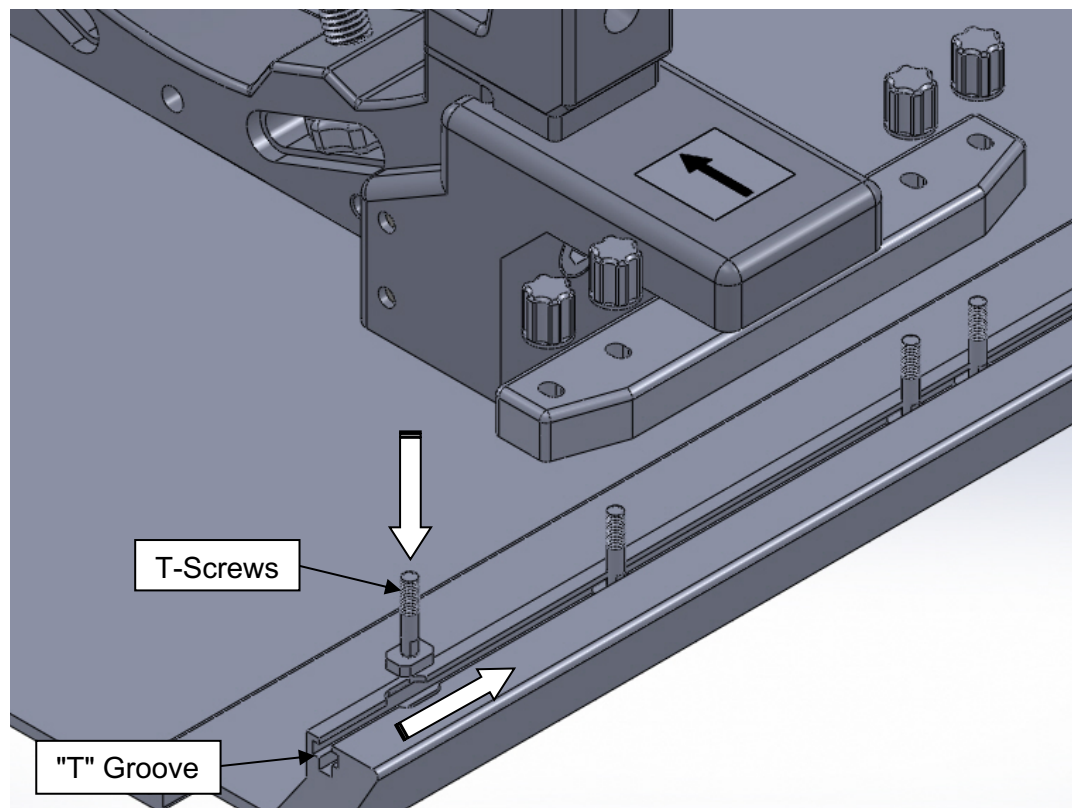

3. Once all 8ea T-screws are aligned with the 2ea T-bars – Lower the Head frame down onto the Cradle.

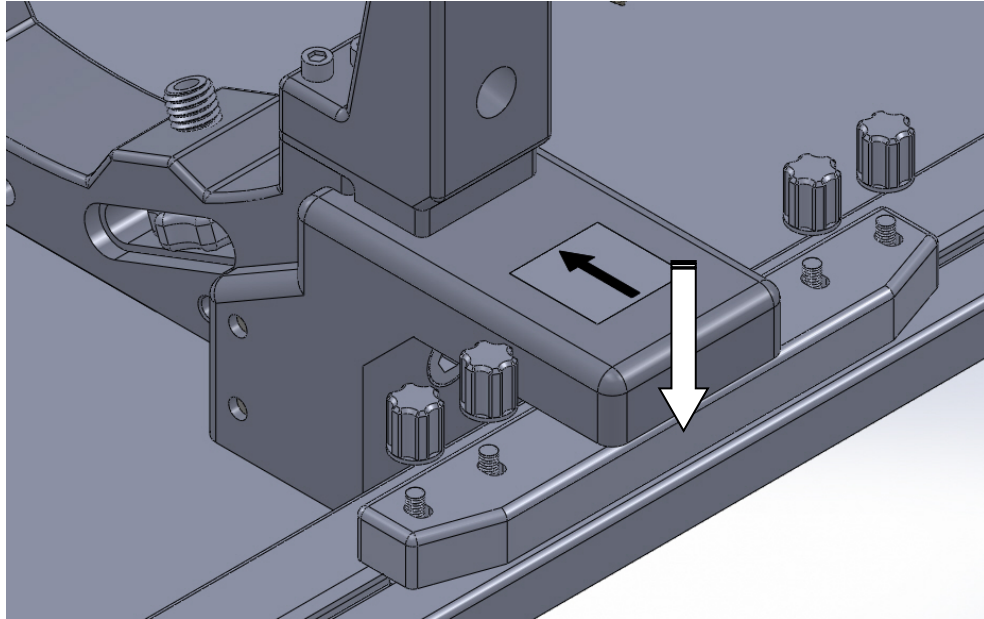4. Fasten the Head Frame to the Cradle using the 8ea Knobs.

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ClearPoint Neuro, Inc.