CLEARPOINT[®]

NEURO

IMRIS Multi-Positional Head Fixation Frame

INSTRUCTIONS FOR USE – INSTALLATION

	<p>NOTE: These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use.</p>
---	---

Table of Contents

- 1. **Device Description**
- 2. **Use Instructions**
 - A. **ORT 100 – Installation into the MRI Unit**
 - B. **ORT 200 – Installation into the MRI Unit**
 - C. **ORT 400 – Installation into the MRI Unit**

	<p>NOTE: These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use.</p>
--	---

1. **Device Description**

Package Contents:

- CP-MPHFF-232-01: ClearPoint IMRIS Universal Multi-Positional Head Fixation Frame**
Head Fixation Frame, Upper Fixation Screws (4), Lower Fixation Screws (4),
Drive Wheels (2)
- CP-MPHFF-242-01: ClearPoint IMRIS Aera-Skyra Multi-Positional Head Fixation Frame**
Head Fixation Frame, Upper Fixation Screws (4), Lower Fixation Screws (4),
Drive Wheels (2)

Associated Devices:

- CP-FFSMPU-01: Upper Fixation Frame Screws (replacements in case of damage)**
Upper Fixation Screws (4)
- CP-FFSMPL-01: Fixation Frame Screws for GE, Siemens and IMRIS (replacements in case of damage)**
Lower Fixation Screws (4)

CP-ITE-01: ORT-200 IMRIS Table Extension

Adaptor for use of ClearPoint Multi-Positional Head Fixation Frame on ORT 200 IMRIS tables.

CP-ITE-02: ORT-400 IMRIS Table Extension

Adaptor for use of ClearPoint Multi-Positional Head Fixation Frame on ORT 400 IMRIS tables.

CP-STB 210-01: CLEARPOINT[®] IMRIS Table Stabilizer

Adaptor for use of ClearPoint Multi-Positional Head Fixation Frames on IMRIS ORT 100 and ORT 200 tables.

NGS-SP-01: Skull Pin Tips

MR Compatible Skull Pin Tips

NGS-SP-02: Pediatric Skull Pin Tips

MR Compatible Skull Pin Tips, Pediatric

The Multi-Positional Head Fixation Frame is designed to immobilize the patient's head during the surgical procedure. The patient's head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure.


The Multi-Positional Head Fixation Frame is compatible with the following IMRIS Surgical Tables:

- IMRIS Surgical Table T1 Series
- IMRIS Surgical Table T2 Series
- IMRIS Surgical Table T2X Series

NOTE:


The images below ONLY show the IMRIS version of the Head Fixation Frame. The IMRIS Aera-Skyra version has a slightly longer tongue on the end but is otherwise the same.

IMRIS Multi-Positional Head Fixation Frame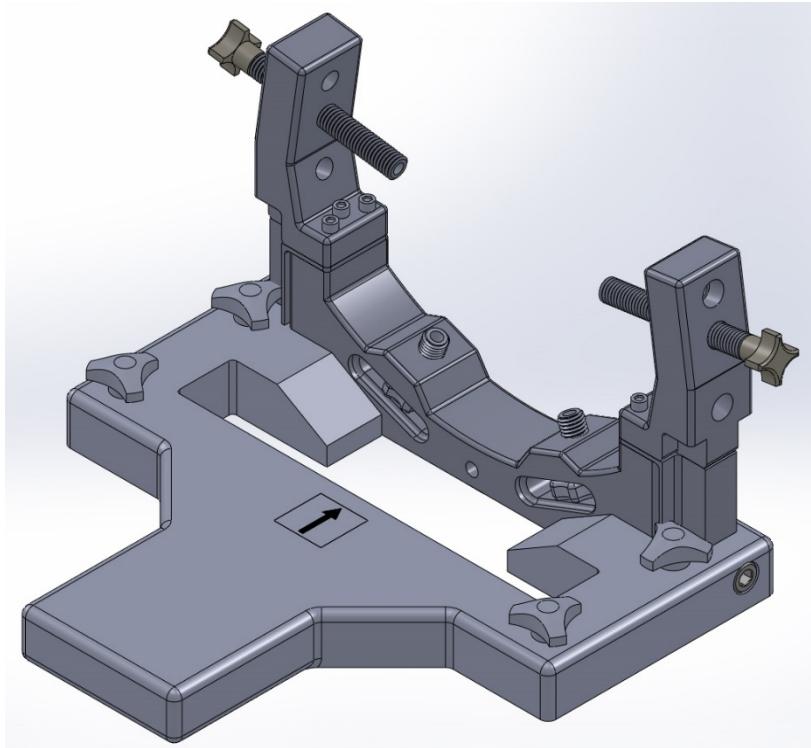

CP-ITE-01: ORT-200 IMRIS Table Extension (*ORT 200 Series Tables ONLY*)


This Table Extension allows mounting the IMRIS Fixed Head Frame onto the IMRIS ORT 200 Surgical Table.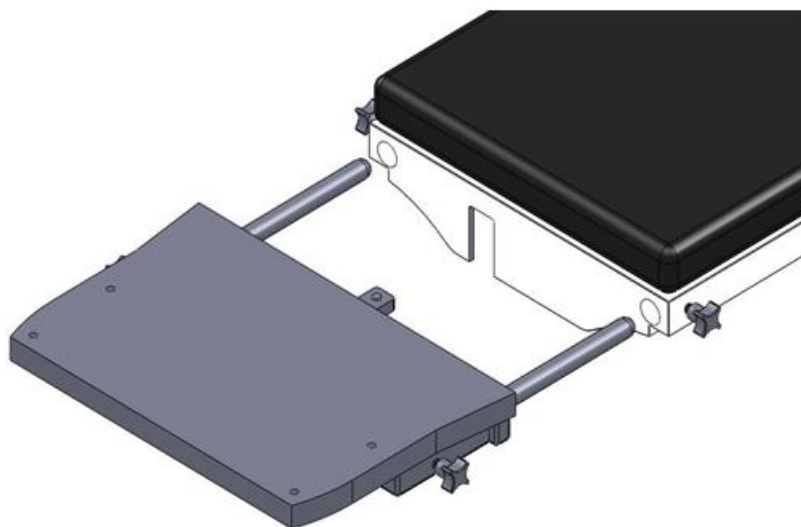

CP-ITE-01 Table Extension (left) with IMRIS ORT 200 Surgical Table (right)

CP-ITE-02: ORT-400 IMRIS Table Extension (*ORT 400 Series Tables ONLY*)


This Table Extension allows mounting the IMRIS Fixed Head Frame onto the IMRIS ORT 400 Surgical Table.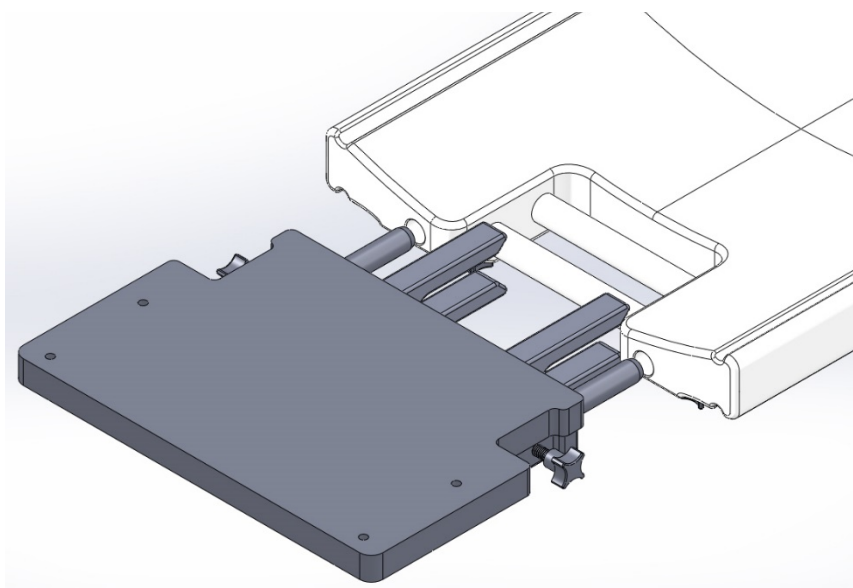


CP-ITE-02 Table Extension (left) with IMRIS ORT 400 Surgical Table (right)

A. ORT 100 – Installation into the MRI Unit:


1. The End Table has 4 ea threaded holes on the top.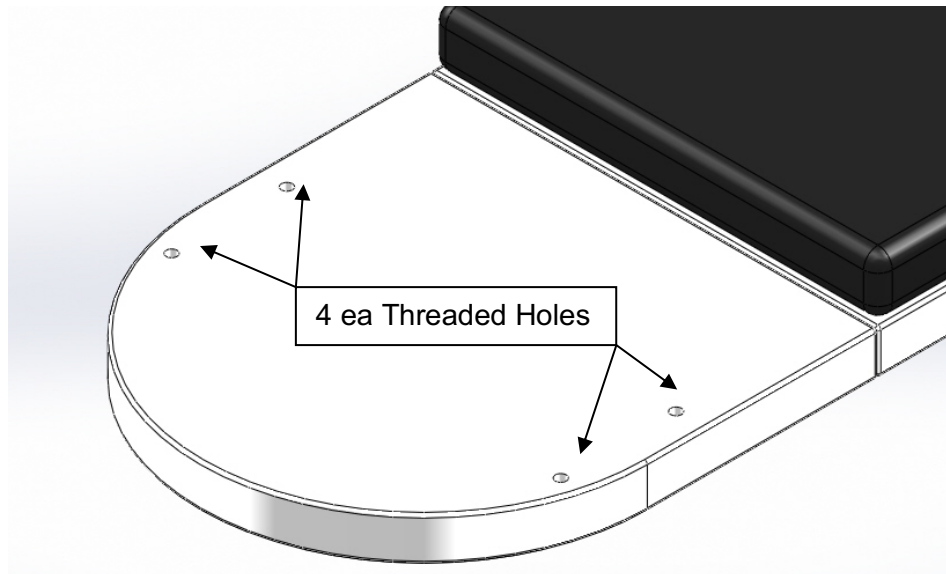

2. Align the 4 ea. Thumbscrews with the holes in the End Table.

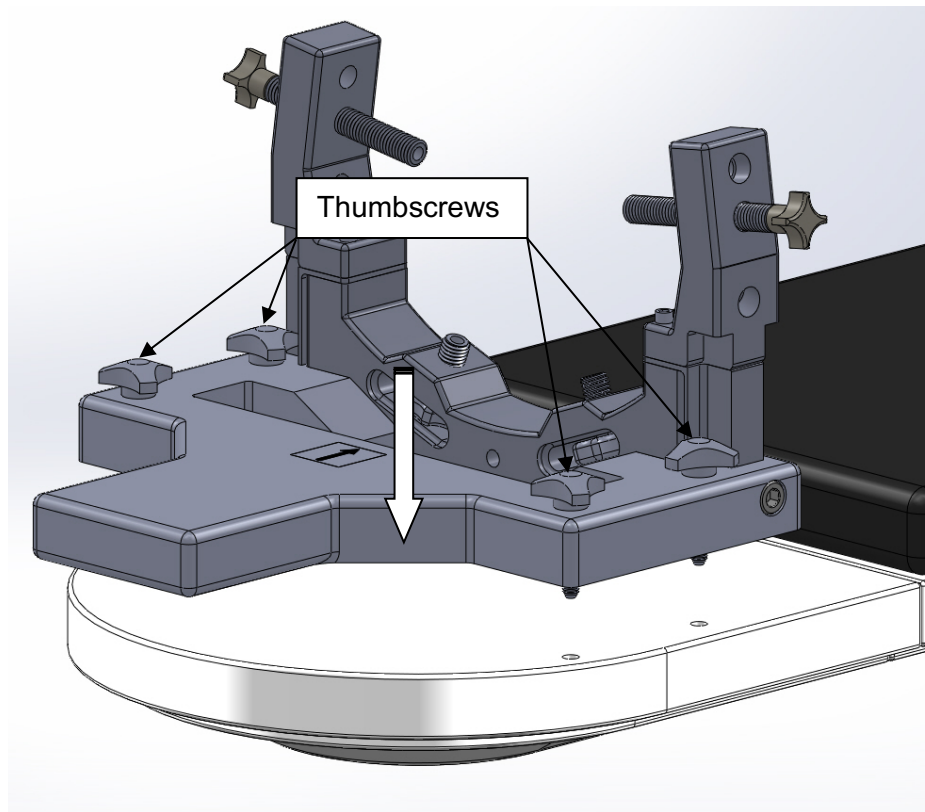
3. Finger tighten the four (4) Table Base Screws to the IMRIS End Table.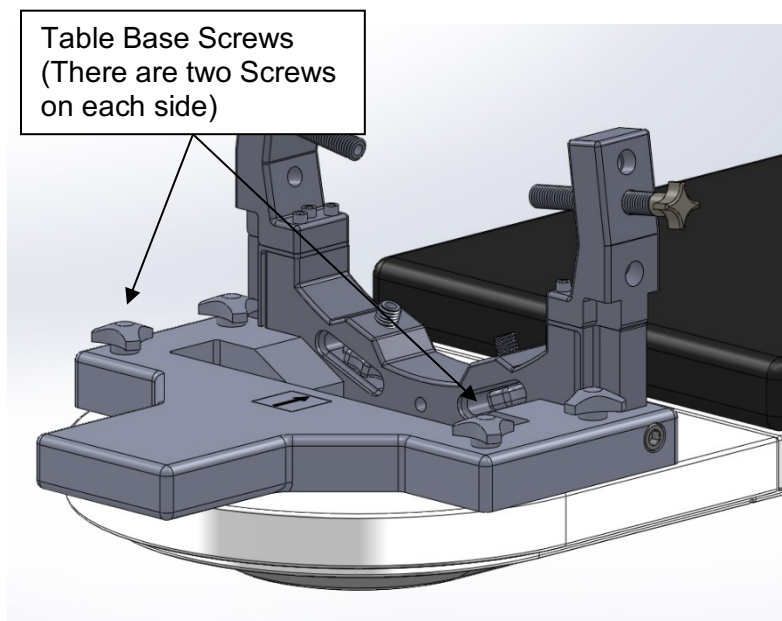


B. ORT 200 – Installation into the MRI Unit:


1. Loosen the two outer Knobs, center Thumbscrew and two inner Thumbscrews on the Table Extension to aid in attachment to the Surgical Table.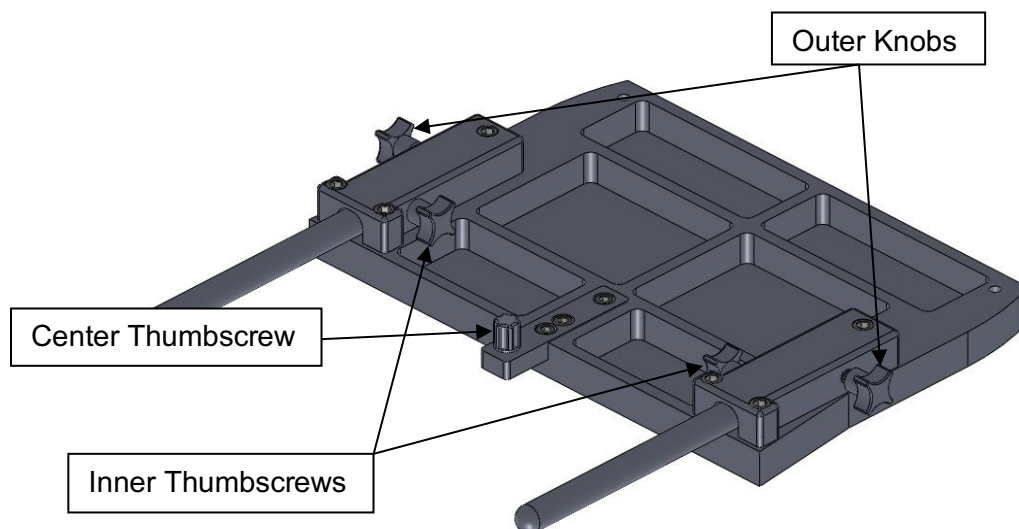


2. Insert the two rods completely into the receiver in the surgical table. A side to side motion may be required during insertion.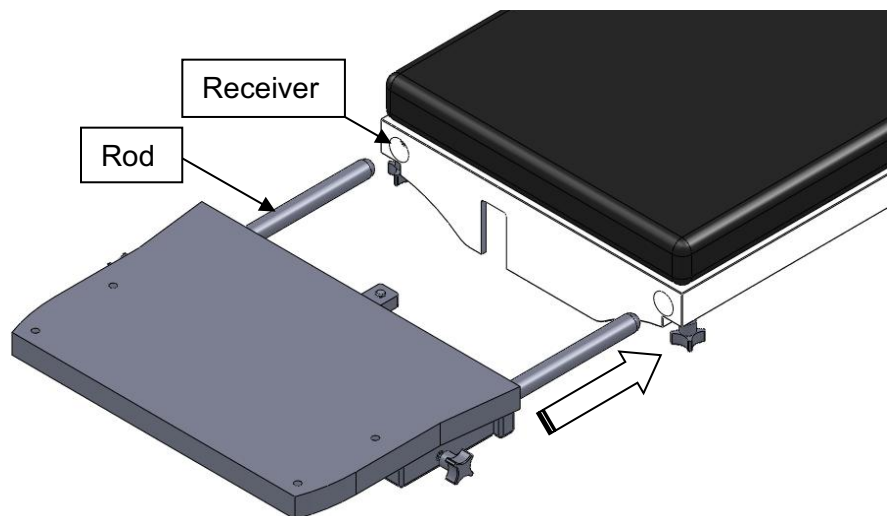


3. Seat the Table Extension firmly against the surgical table.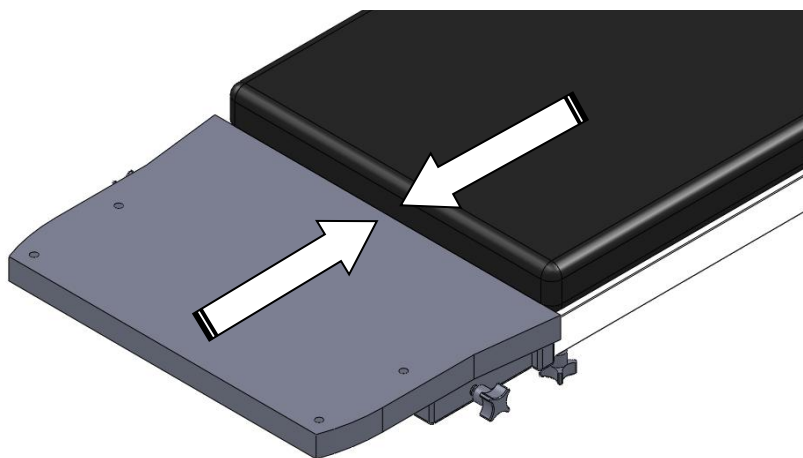


Tighten the two inner Thumbscrews underneath the Table Extension.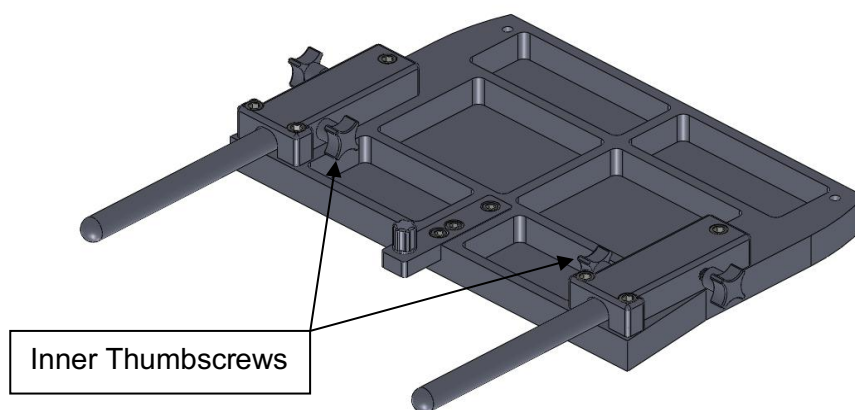


4. Tighten the center Thumbscrew underneath the Table Extension.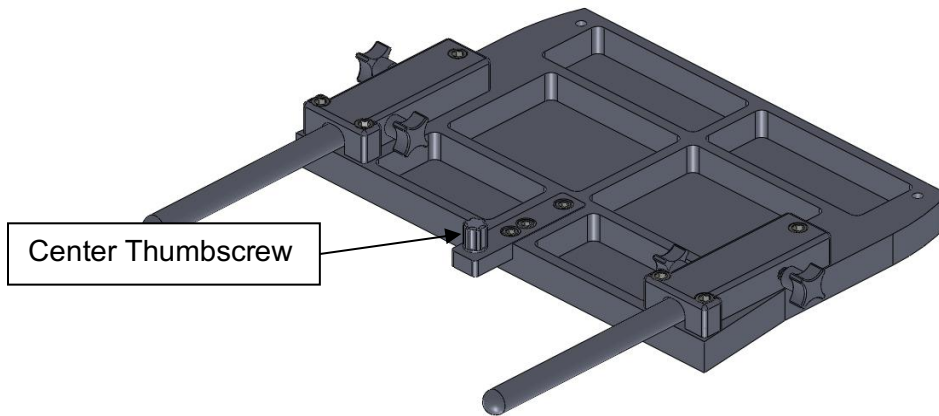


5. Tighten the outer Knobs on the Table Extension .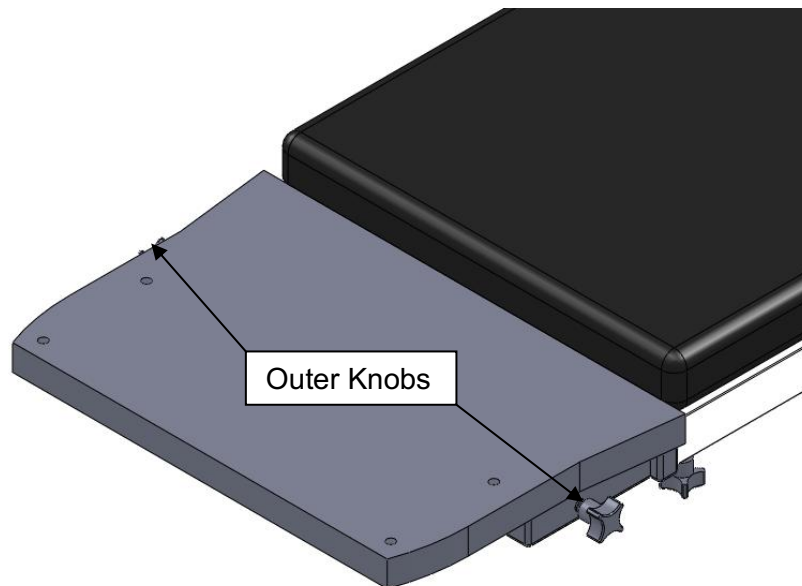


6. Tighten the side Knobs on the Surgical Table to complete the securing of the Table Extension.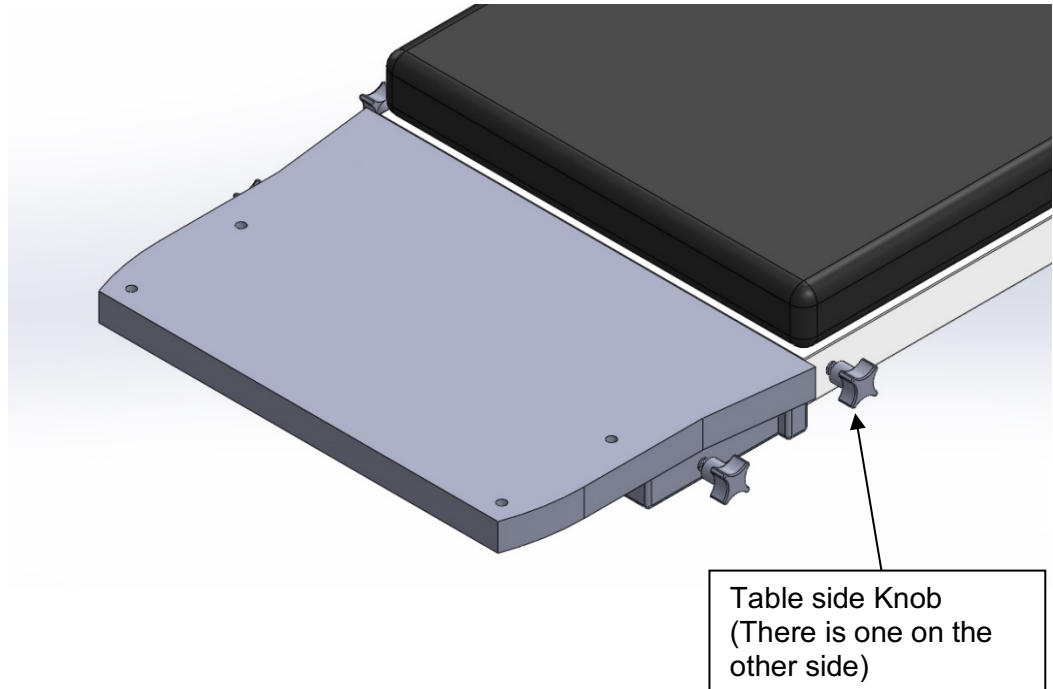

7. Place the Head Fixation Frame assembly onto the ORT 200 Table.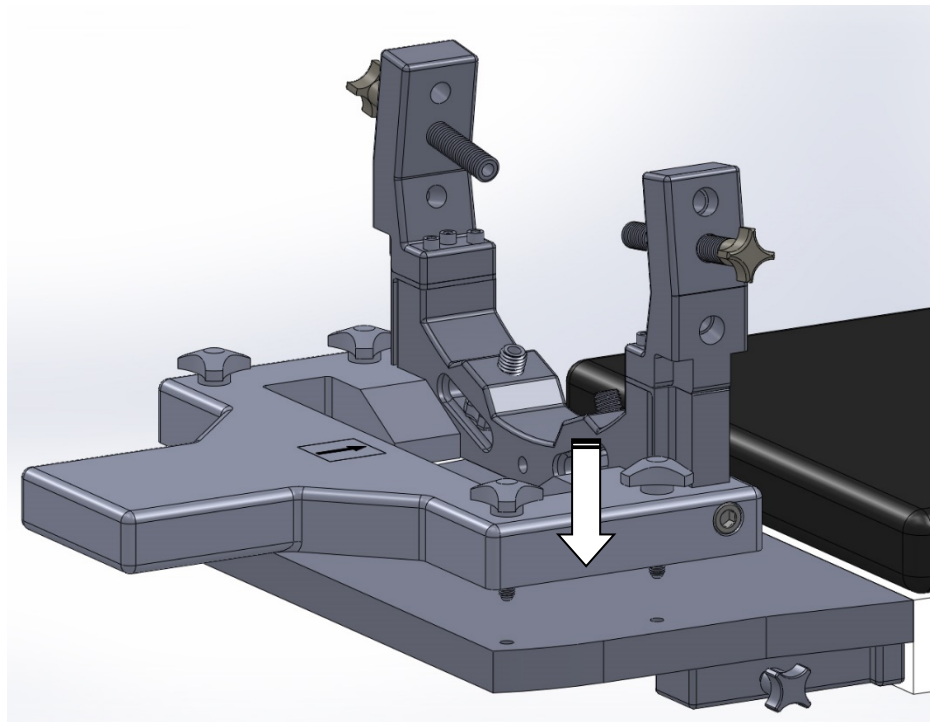



8. Finger tighten the four (4) Table Base Screws to the ORT 200 Table.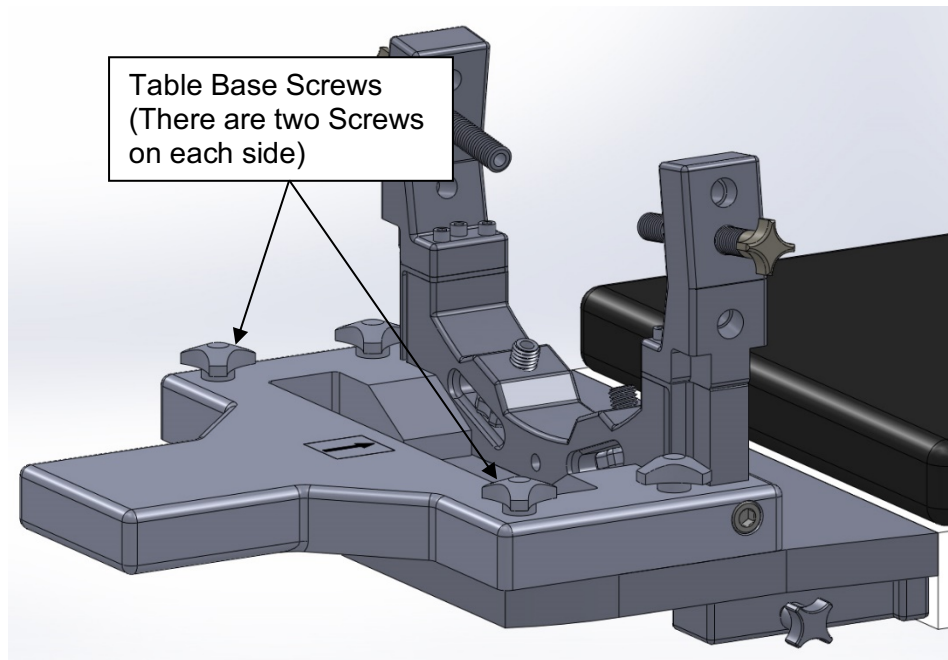


C. ORT 400 – Installation into the MRI Unit


1. Loosen the two Thumb Screws on the Table Extension and the two Thumb Screws on the bottom of the Surgical Table to aid in attachment.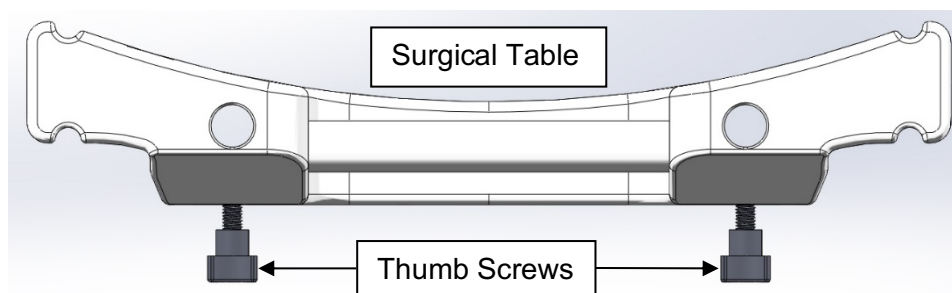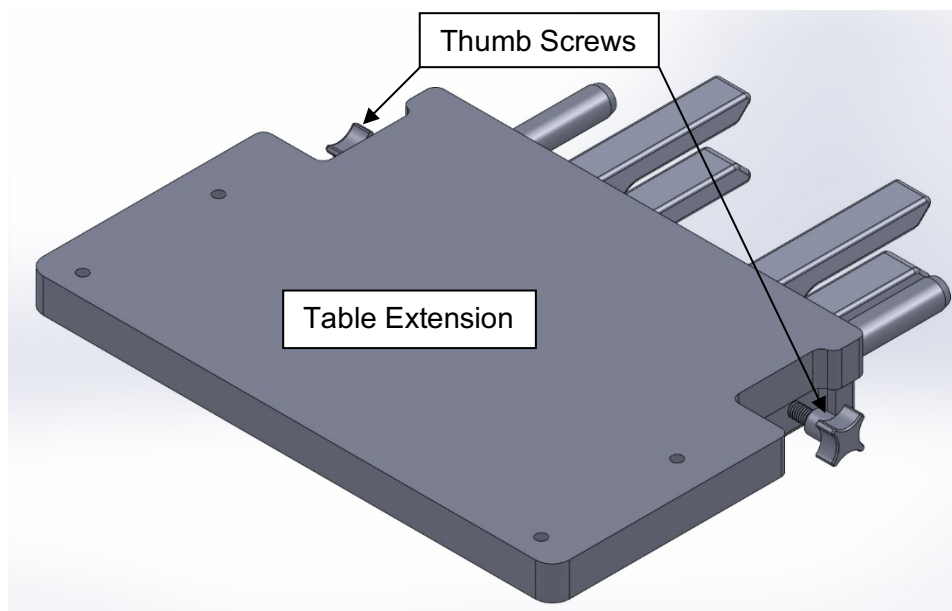


2. Insert the two Rods completely into the receiver in the Surgical Table. A side to side motion may be required during insertion.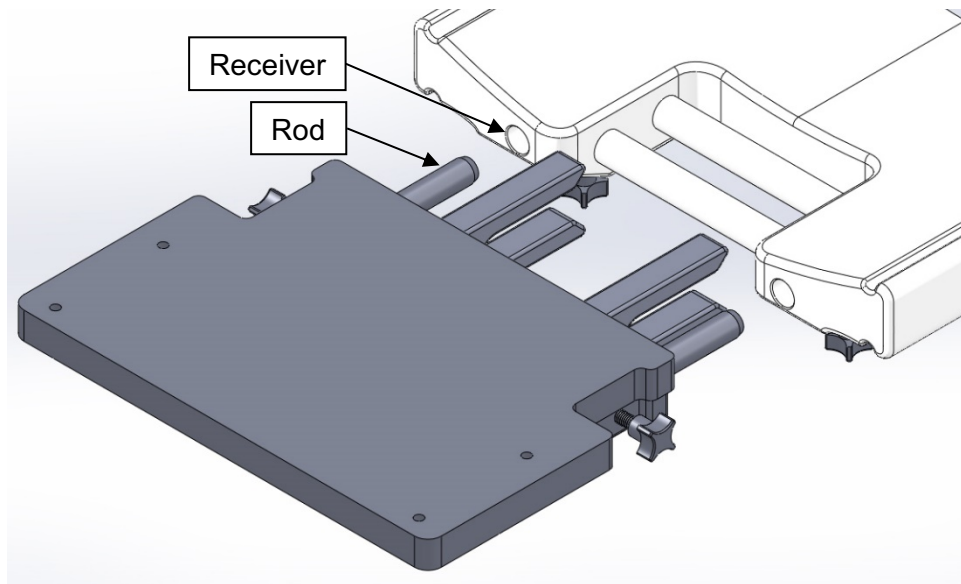


3. Seat the Table Extension against the Surgical Table.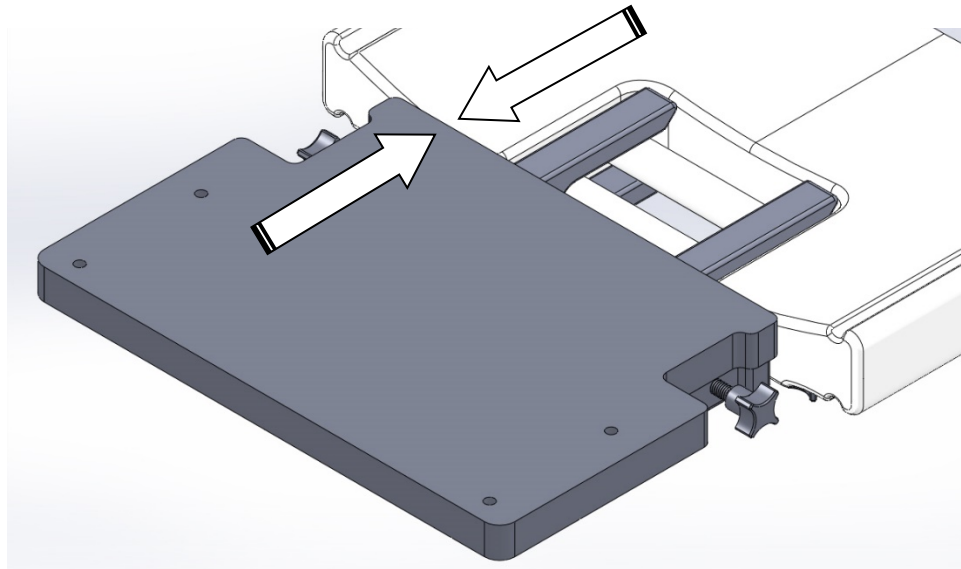


4. Tighten the two Thumb Screws on the Table Extension.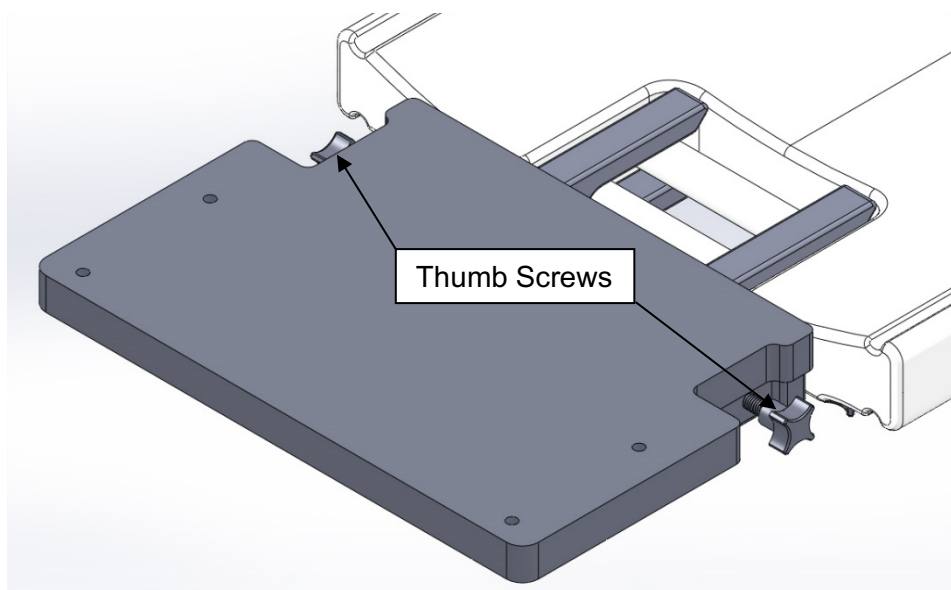


5. Tighten the two Thumb Screws on the bottom of the Surgical Table to complete the securing of the Table Extension.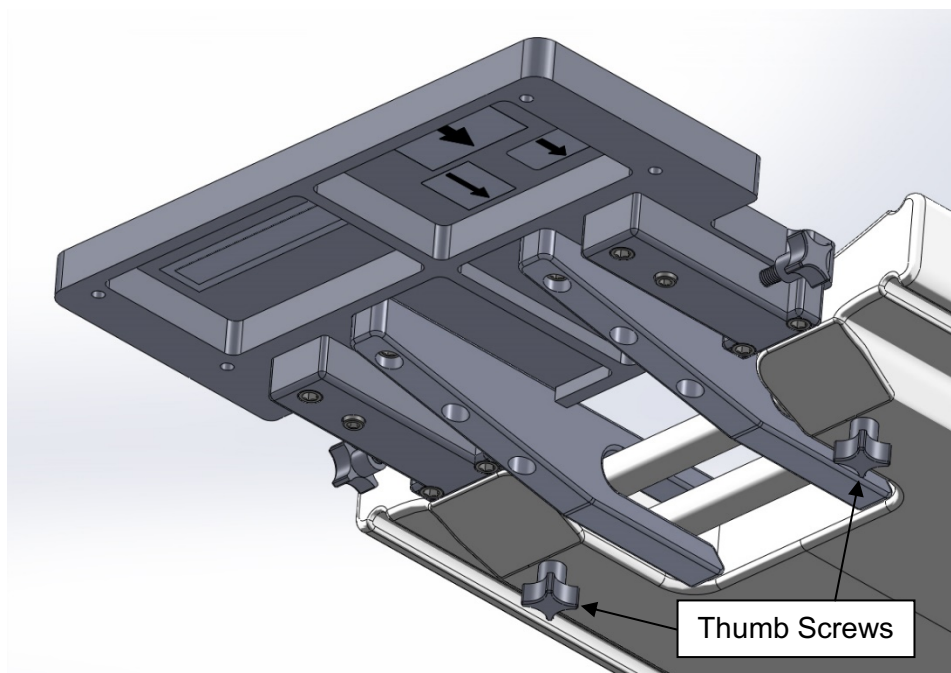

6. Place the Multi-Positional Head Frame assembly onto the ORT 400 Table Extension.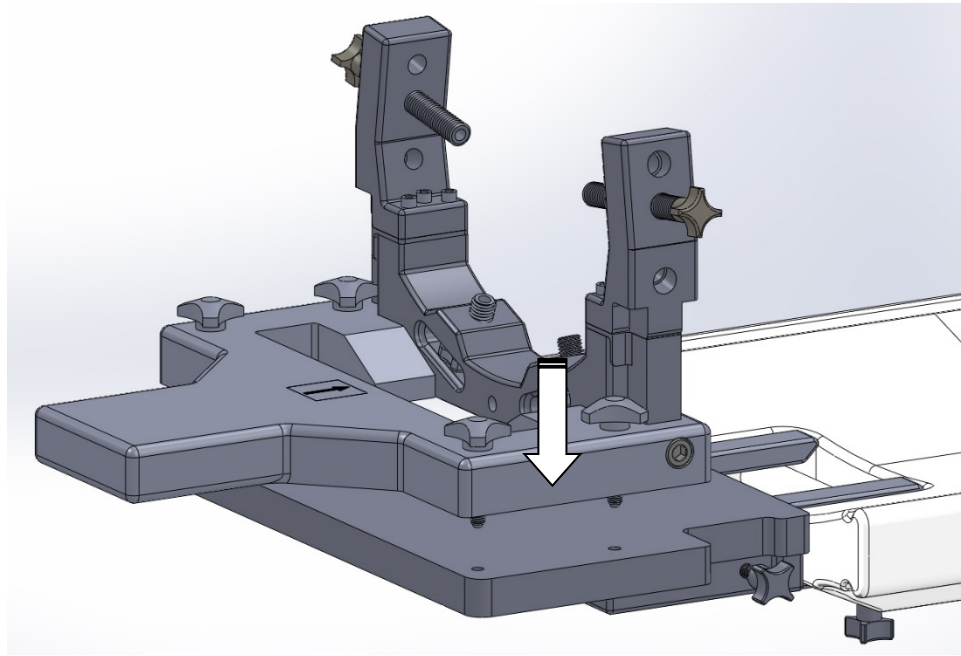

7. Finger tighten the four (4) Frame Base Screws to the ORT 400 Table Extension.

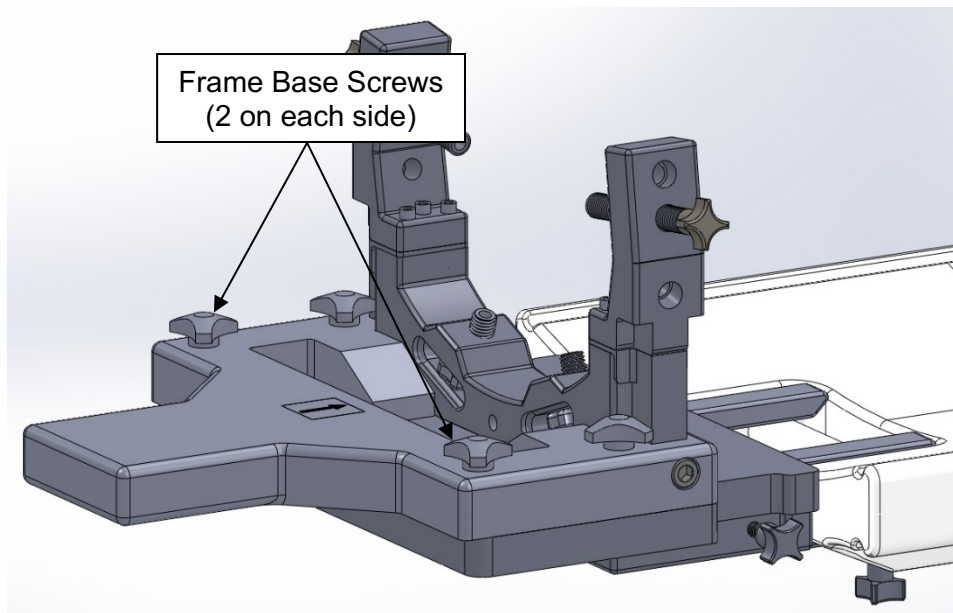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ClearPoint Neuro, Inc.