# CLEARPOINT<sup>®</sup>

## NEURO

**GE-HDw Multi-Positional Head Fixation Frame**

**INSTRUCTIONS FOR USE – INSTALLATION**

**NOTE: These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use.**

## Table of Contents

1. **Device Description**
2. **Use Instructions**
  - A. **Head Fixation Frame Installation**

1. **Device Description**

**Package Contents:**

**CP- MPHFF-322-01: ClearPoint Multi-Positional Head Fixation Frame (New Build)**  
(GE HDw (450w) - MR Table Compatible)

Head Fixation Frame, Upper Fixation Screws (4), Lower Fixation Screws (4), Drive Wheels (2)

**-OR-**

**CP- MPFA-322-01: ClearPoint Multi-Positional Head Fixation Frame (Upgraded)**  
(GE HDw (450w) - MR Table Compatible)

Head Fixation Frame, Upper Fixation Screws (4), Lower Fixation Screws (4), Drive Wheels (2)

**Associated Devices:**

**CP-FFSMPU-01: Upper Fixation Frame Screws (replacements in case of damage)**

Upper Fixation Screws (4)

**CP-FFSMPL-01: Fixation Frame Screws for GE, Siemens and IMRIS (replacements in case of damage)**

Lower Fixation Screws (4)

**NGS-SP-01: Skull Pin Tips**

MR Compatible Skull Pin Tips


**NGS-SP-02: Pediatric Skull Pin Tips**

MR Compatible Skull Pin Tips, Pediatric

The Multi-Positional Head Fixation Frame is designed to immobilize the patient’s head during the surgical procedure. The patient’s head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure.

The GE-HDw Multi-Positional Head Fixation Frame is compatible with the following MRI Systems:


- GE-HDw (450w) 1.5T and 3T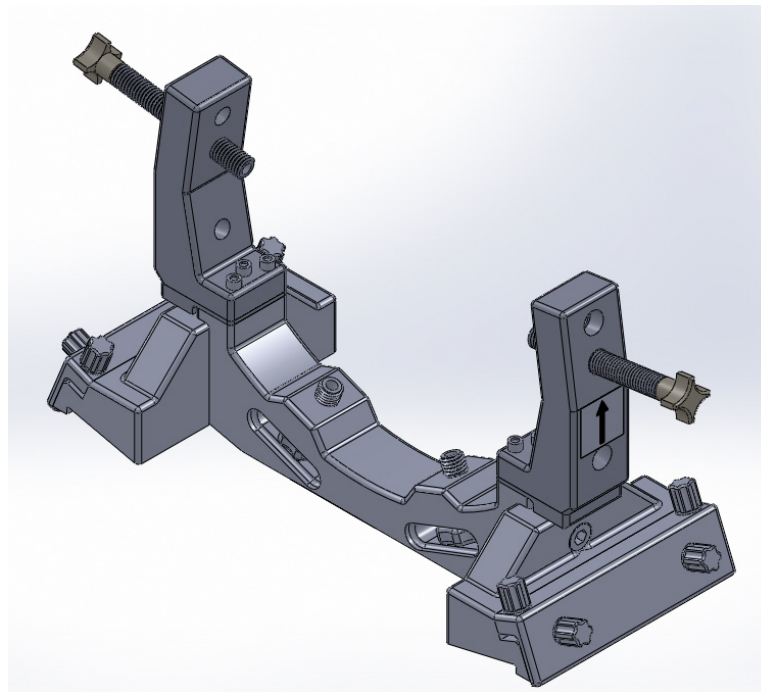


## 2. Use Instructions

### A. Head Fixation Frame Installation


1. The cradle has a groove on each side running the full length of the cradle.
2. At the superior end of the cradle there is a cut-out / pocket on each side of the cradle.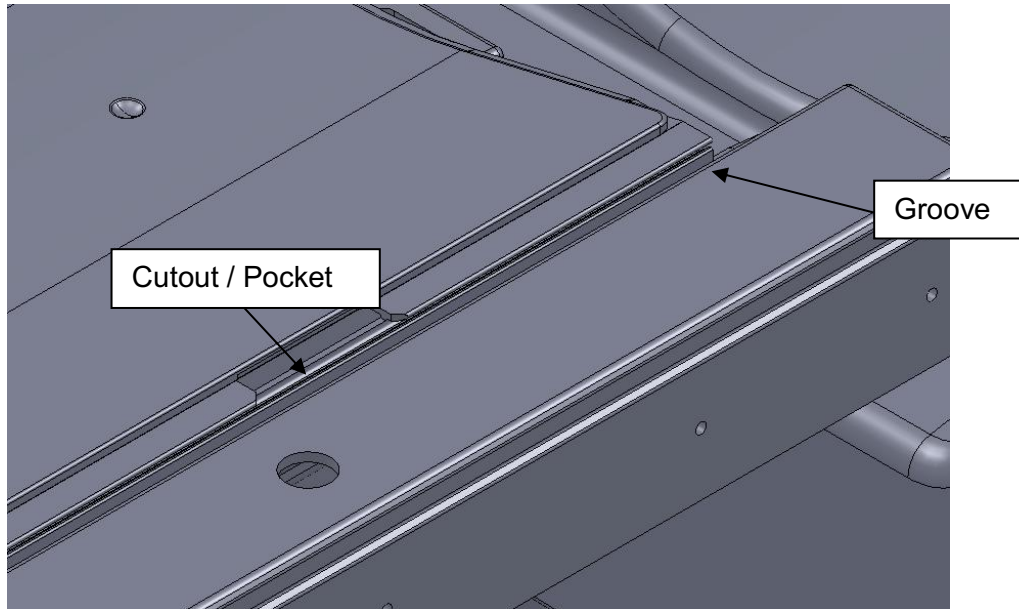


3. Loosen the eight (8) Thumb Screws of the Head Fixation Frame. Four on each side.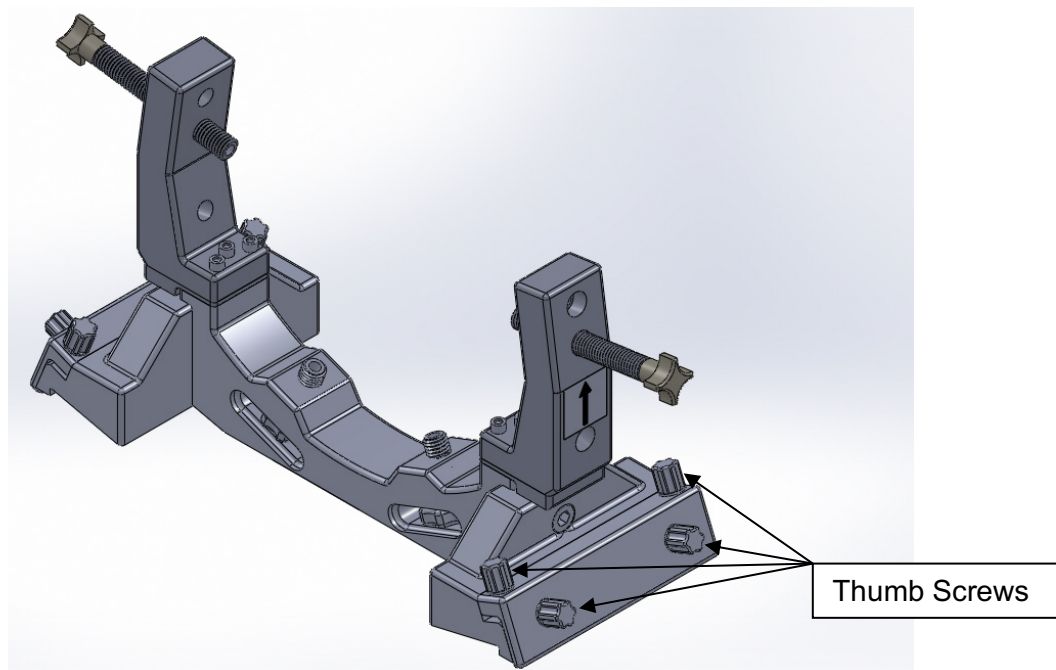


4. The bottom of the Head Fixation Frame has a Toe and a Lip on each side of the Frame.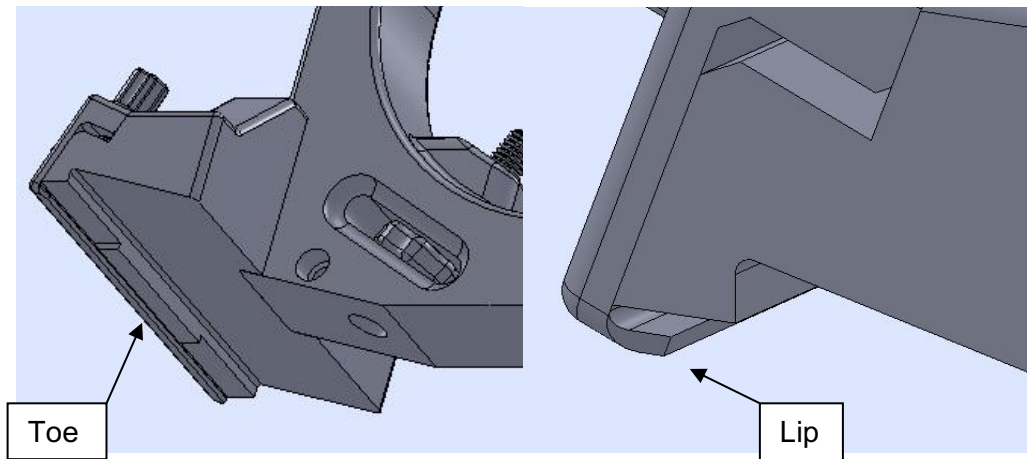

5. Place the Head Fixation Frame onto the cradle with the Toes inside the cut-out / pockets and the Lips into the grooves on each side of the cradle.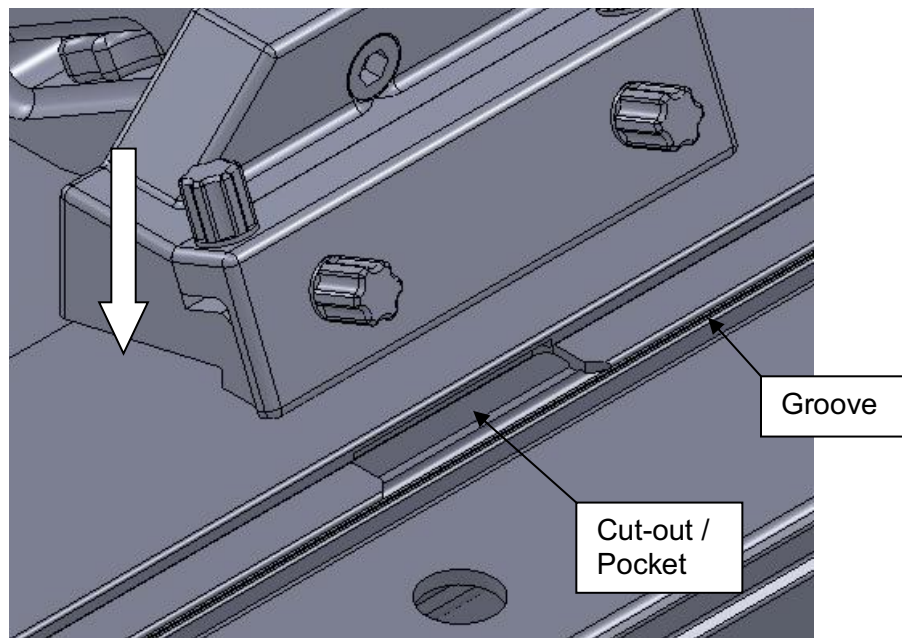

6. 1) Tighten the side Thumbscrews first (leave them slightly loose). 2) Tighten the top Thumbscrews until the Head Fixation Frame is secured to the cradle. 3) Now tighten the side Thumbscrews until the Head Fixation Frame is secured to the cradle. Repeat this on the other side.

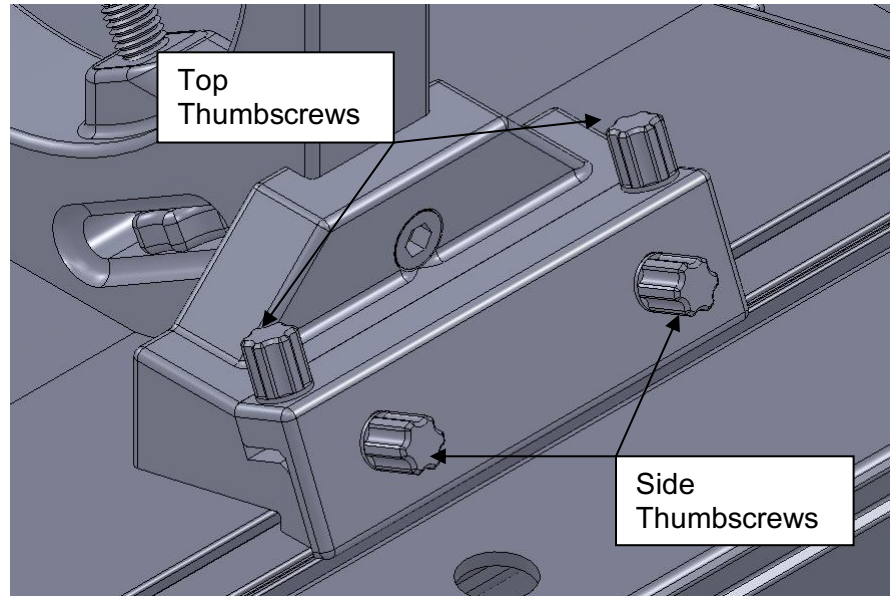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ClearPoint Neuro, Inc.